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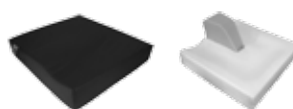
EN

ADVICES

The anti-bedsore cushions prevent the suffering of bed sores and decubitus ulcers. These diseases are totally connected to people who stay motionless over a long period of time. The injuries occur mainly in the body's support points as: heels, buttocks, shoulders, knees, elbows and hips. The excessive pressure of the bones deprives them of blood as well as oxygen which causes first a redness in the zone, then a destruction of the skin tissue and finally the emergence of deep wounds. To avoid the emergence of these symptoms, it would be convenient to place appropriate products behind each support point and to change the patient in different body positions regularly (every 3 hours on average). It's also crucial to avoid any type of humidity because this fact favors the formation of these bed sores.

USER INTRUCTIONS

Depending on the risk of bed sore and user weight, it would be convenient to use one type of cushion or another.

*Bed sore risk evaluation. Norton Scale.

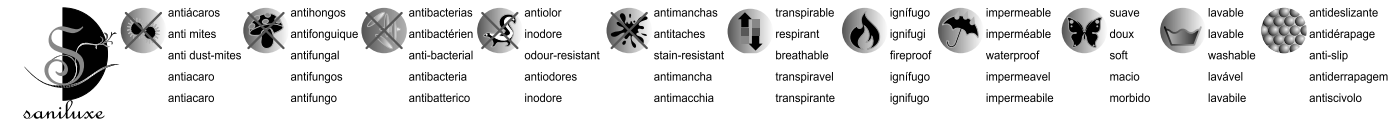
**It's suitable for use both in regular seats (chairs, armchairs, seats, etc.) and in wheelchairs (with or without portable toilet). It also can be used in other means of transportation reserved for people with reduced mobility.

***Custom-made cushion.

****SANITIZED

****SANILUXE

CLASSIFICATION

ANTI-BEDSORE CUSHIONS are considered a Type 1 medical device (rule 1). The conformity evaluation was carried out in accordance with Annex VII (EC Declaration of Conformity) of the Directive 93/42/CEE and the Directive 2007/47.

In accordance with Annex IX of the Directive 93/42/CEE and its subsequent amendments, the sanitary product is defined as:

-Non-Invasive Sanitary Product

Rule 1. – All non-invasive products will be included in the Type I category, unless any of the following rules is applicable.

MODEL/MESURE		REFERENCE	MAXIMUM WEIGHT	DENSITY	INDICATIONS** /COMPOSITION	DEGREE OF RISK*
SANITIZED***	ROUND W/HOLE (white-grey-navy)44X11	101100-101200-101201	75 Kg		Bed sore's prevention, improve the rest. It's recommended for postpartum, hemorrhoids or fistulas. Polyester, terlenka / Polyester siliconized fibre filling.	Low-Medium
	SQUARE W/HOLE (white-grey-navy) 44X44X11	101101-101210-101212				
	ROUND HORSESHOE (white-grey-navy) 44X44X11	101102-101202-101203				
	SQUARE HORSESHOE (white-grey-navy) 44X44X11	101104-101204-101205				
	SQUARE (white-grey-navy) 44X44X11	101103-101106-101108				
SANILUXE****	ROUND W/HOLE (black-white-sky) 44X11	114100-114140-114138	80 Kg		Bed sore's prevention, improve the rest. It's recommended for postpartum, hemorrhoids or fistulas. Polyurethane / Polyester siliconized fibre filling.	Medium
	SQUARE W/HOLE (black-white-sky) 44X44X11	114115-114128-114126				
	ROUND HORSESHOE (black-white-sky) 44X44X11	114105-114136-114134				
	SQUARE HORSESHOE (black-white-sky) 44X44X11	114110-114132-114130				
	SQUARE (black-white-sky) 44X44X11	114120-114124-114122				
MAXICONFORT	SQUARE 42X42X8	102100	95 Kg	50 kg/m3	Bed sore's prevention Polyurethane / Visco-elastic foam	Medium
	HORSESHOE 42X42X8	102300	110 Kg	50 kg/m3 Tatami	Bed sore's prevention Polyurethane/Visco-elastic foam and tatami (5+3 cm)	High
	ROUND W/HOLE 42X8	102160	80 Kg	50 kg/m3	Bed sore's prevention, improve the rest. It's recommended for postpartum, hemorrhoids or fistulas. Polyurethane/Visco-elastic foam and tatami (5+3 cm)	Medium

MODEL/MESURE		REFERENCE	MAXIMUM WEIGHT	DENSITY	INDICATIONS** /COMPOSITION	DEGREE OF RISK*
ERGOPLUS	42X42X8	102405	100/ 120 Kg	90-100 kg/m3	Bed sore's prevention Polyurethane / Visco-elastic foam of high density	High
	45X42X8,5	102407				
	W/ ABDUCTOR PLUG 42X42X8	102408	100 Kg	90 kg/m3	Bed sore's prevention, anti-slipping Polyurethane / Visco-elastic foam of high density and foam plug	
ERGO TECH	42X42X10	102805	140 Kg	100-120 kg/m3	Bed sore's prevention, anti-slipping Polyurethane / Visco-elastic foam of high density and foam plug	High
	45X42X10	102810				
PREVENTIVE ***	50x50x10	102200	140 kg	60 kg/m3 Tatami	Bed sore's prevention Polyurethane / Visco-elastic foam and tatami (7+3 cm)	High
	WITH DISPLACED HOLE 42x42x8	102250	100 Kg		Bed sore's prevention, improve the rest. It's recommended for postpartum, hemorrhoids or fistulas. Polyurethane / Visco-elastic foam and tatami (5+3 cm)	
BASIC	41X39X6,5	102702	90 kg	80 kg/m3	Bed sore's prevention Polyurethane / Visco-elastic foam of high density	High
	ROUND WITH HOLE 41x6,5	102450			Bed sore's prevention, improve the rest. It's recommended for postpartum, hemorrhoids or fistulas. Polyurethane / Visco-elastic foam of high density	Low
ERGOGEL COMPLEX 42X42X8		102428	120 kg	90 kg/m3	Bed sore's prevention. Polyurethane / Visco-elastic foam of high density and silicone. The coccyx area has a double density.	High
LIQUID FLOATING	SQUARE 42X42X6	103304	110 kg		Bed sore's prevention Polyurethane / PVC, foam, propylene glycol	Medium
	SQUARE 45X42X6	103306				
	HORSESHOE 42X42X6	103302				
SILICONE	42X42X3	103350	120 kg		Bed sore's prevention Polyurethane / silicone gel	Medium
	45X42X3	103352				
AIR CUSHIONS	1 o 2 VALVES Heights: 5/7,5/10 Width x long: 40/45	103600 hasta 103658	120 kg		Bed sore's prevention Polyurethane and anti-slipping base / EVA material	Very high

ANTIBEDSORE CUSHIONS

EN

INSTALLATION

The different forms of the cushions enable the patients to adapt their body positions to any seat as: wheelchairs, armchairs, etc.

It's convenient to place the cushion preferably on a flat base.

The SANITIZED & SANILUXE cushions have straps which serve to hold the cushion to the seat to prevent it from moving when necessary.

The LIQUID FLOTATION, SILICONE & AIRcushions include a carrying handle for easy use.

(The use of the products must be carried out on a flat and rigid surface. In this way, the useful life of the product will be notably prolonged). Another thing to keep in mind is that the non-slipping base of these cushions must be in contact with the seat.

The AIR cushions carry one or two valves depending on the product model, this fact enables to regulate the firmness of the cells throughout the emptying and refilling of air. In this regard, these cushions include an air pump and also a repair set (one bottle of glue). In case of breakage, expand the liquid with the applicator all over the break and leave to dry about three hours. After this, the process has to be repeated. In this way, the cushion will remain fixed. No additional patches. The air cushion is delivered fully inflated. The correct pressure of the cushion will be tested with the user (the person has to sit on the cushion). The buttock has to remain at a distance of 1 cm approximately from the seat cushion base. It's enough to put one finger between the user's buttock and the seat cushion base to test the inflation of this one.

To inflate or deflate the cushion:

- 1.- Unscrew the valve protector.
- 2.- Unscrew the valve rotating it counterclockwise.
- 3.- Tighten to deflate or use the air pump.
- 4.- Screw the valve.
- 5.- Screw the protector.

CLEANING/SANITIZING

The majority of cushions have the SANITIZED's cover, except for the SANILUXE cushions.

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SANITIZED cushions:

Brush or shake the surface to ensure the product in good shape and slow its deterioration. Wash in cold water. Do not use abrasive products. Machine washable (use a delicate cycle and suppress the centrifugation). It's recommended to use a special cover to avoid an excessive deformation during the washing. Leave to dry in a horizontal position. Do not iron. Do not use bleach.

Cleaning and sanitizing process for the cover of the rest of cushion models:

Hand and machine washable cover by using neutral detergent at 90°C. Do not use centrifugation. Dry cleaning is possible. Use drier at a low temperature. Do not iron. Do not use bleach.

IMPORTANT: DO NOT WET THE MEMORY FOAM.

ANTIBEDSORE CUSHIONS

EN

SECURITY

Test the correct adjustment of the cushion to the seat where is going to be used (the cushion has to be about 1 or 2 cm smaller than the seat).
According to the AIR cushion's valve: DO NOT USE TOOLS AS WRENCHES OR PLIERS WHEN TO OPEN OR CLOSE THE VALVES. OPEN AND CLOSE ONLY AND EXCLUSIVELY WITH THE FINGERS.

GENERAL PRECAUTIONS

Do not use caustic, abrasive, solvent or sharp objects in direct contact with the cushion.
Test regularly the cushion deterioration (orifices, cracks, surface's modification, etc.)

MATERIAL'S PRECAUTIONS

Avoid prolonged exposure to liquids and humidity conditions of the cushion. Do not expose to direct sunlight.
A visual control of the general cushion's status must be regularly carried out: the presence of material deformations or a inhomogeneous and slow return are considered as aging principles that put the support proprieties of bed sores protection at risk.

MAINTENANCE

Store the cushions preferably in a flat position and protect against sunlight and humidity.

PRODUCT RECYCLING

Do not burn or throw away in the nature, outside the correct places for this action.

GUARANTEE

The guarantee period of all the cushions is two years from the date of the original purchase (except for liquid flotation and silicone cushions which have a 3-month guarantee).
This guarantee does not cover the regular deterioration of the product and does not substitute legal guarantees.
Please contact with the Quality Department of the company to any incident that may occur to resolve it.

LIFETIME OF THE PRODUCT

The regular lifetime of the cushions is about a year.

COMPOSITION

See "user's instructions" section.

LEGEND AND LABEL SYMBOLS

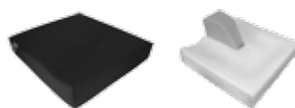
	Maker.
LOT	Serial number.
REF	Catalog number.
	Read the instructions for use first.
	Certificate of conformity with Directive 93/42 / EEC
	Caution. Read the instructions for use first.

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