EVOLUXION

JUST FOR YOU!

ALMOFADAS ANTI ESCARAS – MANUAL DE UTILIZAÇÃO
COJINES ANTIESCARAS – MANUAL DE USUARIO
ANTIDECUBITUS CUSHION – USER MANUAL

COUSSINS ANTI-ESCARRES – MANUEL UTILIZATEUR
CUSCINI ANTI-ULCERA – MANUALE UTENTE
ANTI-DEKUBITUS KISSEN – BENUTZERHANDBUCH

Almofadas Anti Escaras:

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Antidecubitus Cushion:

The antidecubitus cushion prevent the appearance of pressure ulcers, consequence from the permanence of the body in the same position for long periods of time. Pressure ulcers appear mainly in areas of bony prominence and support points of the body, such as heels, gluteus, shoulders, knees, elbows and pelvis. Excessive pressure on a point of the body, limits the intake of blood to that area and consequently oxygen, causing redness, destruction of the dermis/epidermis and consequently superficial or deep wounds.

To avoid the appearance of pressure ulcers, caregivers should use different solutions: Regularly change the positioning of the body; Use cushions or devices that minimize pressure at each point of support; Use moisturizing skin creams.

Model Selection and Usage Indications:

According to the risk of Pressure ulcers (Braden scale or Norton) and weight of the user.

Evolution seating cushions are suitable for both wheelchairs and conventional seats, such as sofas, chairs, seats, etc.

Classification:

Evolution seating cushions are considered Class I medical devices, in accordance with Directive 93/42/EEC.

Nantong Yeming Textile Technology Co., Ltd
Building 1, Group 20, Tangfang village, Zhangzhishan Town
Tongzhou District, Nantong City, Jiangsu.

Kingsmead Service B.V.
Zonnehof 36, 2632 BE, Nootdorp, Netherland
office@kingsmead-service.com

Model	Dimensions	REF.	Max weight user	Indications and composition	Degree of Risk*
EvoGel	42X42X8 cm	EV700	120 kg	Prevention of Pressure Ulcers. Polyurethane/Viscoelastic high density and silicone. The coccyx area contains double density Avoid tension forces on the cushion surface	HIGH
EvoTech	42X42X8 cm	EV900	100kg	Prevention of Pressure Ulcers. High density Viscoelastic foam	MEDIUM/HIGH
EvoVisco	42X42X8 cm	EV800	90kg	Prevention of Pressure Ulcers. High density Viscoelastic foam	MEDIUM/HIGH

* - Braden or Norton Tables

Cleaning and Maintenance:

Cover:

All cushions bring a removable cover, which allows you to wash. It can be washed by hand with cold water, using non-abrasive products or can be washed in the machine, using a program for delicate, cold and centrifuge-free clothing. It is recommended to use a safety net to avoid deformation of the cover during washing.

Dry in the Horizontal position.

Do not use bleach.

Don't iron.

Cushion Core: Viscoelastic foam block - Do not **wet**.

Safety:

Make sure that the cushion fits the seat properly.

Storage:

Store the cushion preferably horizontally, avoiding sun exposure and moisture.

Precautions:

- Avoid contact of the cushion with solvents, corrosive products and sharp objects.
- Regularly check the normal wear and tear of the cushion, such as appearance modifications, possible snares and/or holes.
- Avoid prolonged contact with liquids.
- Avoid sun exposure;
- Avoid moisture;
- Regularly carry out a visual control of the condition of the cushion, as the aging and wear of the cushion compromises the properties of the cushion, and consequently its function of preventing pressure ulcers.

Warranty:

Period of 2 years from the date of purchase. The warranty does not cover the normal wear and tear of the product, as well as inappropriate use of the product. For complaint, contact the point of sale where the cushion was purchased, being mandatory, the presentation of proof of purchase with date and the information of the Article Lot.

Symbols and Legend:

	Attention. Read the Instructions first
	Manufacturer
LOTE	Lot
REF	Reference within the EvoluXion Range
	Read the Instructions first.
CE	Certified in accordance with Directive 93/42/EEC

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