Dear Customer, thank you for the trust placed in EMO product. Picking the line you have acquired a quality product and high level sanitary doctor. Read these instructions carefully. If in doubt contact your doctor or specialist dealer.

Regulations
The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

Description and characteristics
Ankle Foot Lifter LIF100 Support for the treatment of clubfoot in flaccid paralysis without deviation in varus. The constant evolution of materials and technologies have facilitated the design of Lifter combining design, simplicity, functionality and efficiency. Versatile: flaccid paralysis with and without varus deviation. Lightweight foamized and highly breathable material. Minimum thickness: locking and micro-adjustment by velcro. Sizes: Foot-Lifter LIF100 TIBIAL SUPPORT Ankle Foot: S: 14-18 M: 18-22 L: 22-26 XL: 26-30. Contour ankle. Color: black.

Components: Foot-Lifter LIF100 consists of 4 components: - tibial Ankle Foot Support: clamp elastic adjustment to suit the patient's anatomy. It incorporates a viscoelastic pad on the Achilles tendon and the other on the instep flex zone. - LIF131 Lifter pull tab: Is inserted between the tongue and shoe Cordonera. - Elastico Lifter LIF132 strength: maximum strength elastic cord connecting the tibial traction and support and regulating the degree of foot dorsiflexion of the patient. - LIF133 control Elastic: elastic cord for maximum strength that compensates the deviation in varus. Support LIF100 Tibial Foot Ankle Foot Lifter consists of: - tibial Ankle Foot Support. - Support LIF131 Traction Lifter (2 units). - Elastico Lifter Traccion: LIF132 LIF133 Elastico Lifter Control. Lifter LIF141 traction hooks (6 pcs).

Instructions for use and installation
The product must be adapted by the prosthetist or by the physician individually for each patient. The right choice of size is vital for maximum treatment efficacy and prolong the life of the garment. LIF100: Enter the LIF131 Lifter pull tab between the tongue and the shoe Cordonera. Then adjust the tibial Ankle Foot Support help of two independent straps. Place the brace on the area to the correct height for use, placing the pad viscoelastic profile on the Achilles tendon and the other on the instep flex zone. Elastic Lifter pass the traction LIF132 through LIF131 Lifter tensile support and then through the tibial Ankle Foot Support guides. Adjust the Velcro end regulating dorsiflexion. Finally, attach the clamp LIF133 Elastic shoe and pass control to the elastic through the loop / guide and regulate. In the case of using the forefoot support Lifter LIF101 it replaces the support Lifter LIF131 traction. When using tow hooks LIF141 Lifter: Enter the hooks for shoe eyelets and pass them LIF132 Lifter elastic traction. LIF500 Lifter Lifter LIF600 brachial LIF600 forearm brachial Lifter Orthotic device comprising a support shoulder pad comprising a forearm support and joined by a set of straps with Velcro micro-adjustment means. The pad includes a core of rigid plastic to improve the function of the weight of the braço. La traction / suspension arm support is pro-vived by two cinchas. Adicionalmente has a strap positioning control to position the arm and shoulder in the desired position. It allows placement of the brace by the patient with a single hand. The design facilitates the patient bracing maximum functionality and freedom of movement of the arm. Made with a breathable soft. S (23 cm - 28 cm.); M (28 cm - 34 cm). L (34 cm - 40 cm). (Biceps contour)

LIF600 forearm brachial Lifter Orthotic device comprising a support shoulder pad comprising a forearm support and joined by a set of straps with Velcro micro-adjustment means. The pad includes a core of rigid plastic to improve the function of the weight of the braço. La traction / suspension arm support is provided by two straps. Additionally it has a control rod positioning to place the arm and shoulder in the desired

position. Specially designed for patients who maintain some control of mobility biceps. Allows placement of the brace by the patient with one hand.

The design of the brace facilitates the patient maximum functionality and freedom of movement of the arm. Made with a breathable soft. S (23 cm - 28 cm.); M (28 cm - 34 cm). L (34 cm - 40 cm). (Biceps contour)

Effects LIF500 and LIF600

It supports the weight of the arm getting shoulder joint coaptation. Stabilizes the shoulder joint and freezing controlling functionality subluxation and internal rotation. Previene reset and the capsular and tendons of the shoulder joint stretching, avoiding muscle and nerve problems. Compensates regulates internal rotation and external rotation desired thanks to tight control of positioning. Prevents pain and joint subluxation.

Indications

Ankle Foot Lifter LIF100 Support: equino foot in flaccid paralysis with and without varus deviation. LIF500 Lifter Lifter LIF600 brachial brachial and forearm: hemiplegia after stroke, neurological upper limb injuries, brain trauma skull, brachial plexus injury with paraparesis and high breathable material. Minimum thickness: locking and micro-adjustment by velcro. Sizes: Foot-Lifter LIF100 TIBIAL SUPPORT Ankle Foot: S: 14-18 M: 18-22 L: 22-26 XL: 26-30. Contour ankle. Color: black.

Components: Foot-Lifter LIF100 consists of 4 components: - tibial Ankle Foot Support: clamp elastic adjustment to suit the patient's anatomy. It incorporates a viscoelastic pad on the Achilles tendon and the other on the instep flex zone. - LIF131 Lifter pull tab: Is inserted between the tongue and shoe Cordonera. - Elastico Lifter LIF132 strength: maximum strength elastic cord connecting the tibial traction and support and regulating the degree of foot dorsiflexion of the patient. - LIF133 control Elastic: elastic cord for maximum strength that compensates the deviation in varus. Support LIF100 Tibial Foot Ankle Foot Lifter consists of: - tibial Ankle Foot Support. - Support LIF131 Traction Lifter (2 units). - Elastico Lifter Traccion: LIF132 LIF133 Elastico Lifter Control. Lifter LIF141 traction hooks (6 pcs).

Washing and maintenance
Hand wash in warm water (maximum 30 °) and neutral soap. During washing, if the garment has Velcro, they must be glued together or keep them closed. Dry at room temperature. Do not store or iron the garment. Contact with ointments, creams, alcohols and solvents may damage the garment.

Quality Certificates
All materials used have been tested, comply with applicable European standards are of high quality and offer maximum safety and comfort during use. Öko-Tex Standard 100: Textile Eco-label awarded after the completion of a catalog of criteria for analysis that guarantees the absence of substances harmful to health. Certifies the absence of harmful substances in all materials that a garment is made.

Instructions for use and installation
The product must be adapted by the prosthetist or by the physician individually for each patient. The right choice of size is vital for maximum treatment efficacy and prolong the life of the garment. LIF100: Enter the LIF131 Lifter pull tab between the tongue and the shoe Cordonera. Then adjust the tibial Ankle Foot Support help of two independent straps. Place the brace on the area to the correct height for use, placing the pad viscoelastic profile on the Achilles tendon and the other on the instep flex zone. Elastic Lifter pass the traction LIF132 through LIF131 Lifter tensile support and then through the tibial Ankle Foot Support guides. Adjust the Velcro end regulating dorsiflexion. Finally, attach the clamp LIF133 Elastic shoe and pass control to the elastic through the loop / guide and regulate. In the case of using the forefoot support Lifter LIF101 it replaces the support Lifter LIF131 traction. When using tow hooks LIF141 Lifter: Enter the hooks for shoe eyelets and pass them LIF132 Lifter elastic traction. LIF500 Lifter Lifter LIF600 brachial LIF600 forearm brachial Lifter Orthotic device comprising a support shoulder pad comprising a forearm support and joined by a set of straps with Velcro micro-adjustment means. The pad includes a core of rigid plastic to improve the function of the weight of the braço. La traction / suspension arm support is pro-vived by two cinchas. Adicionalmente has a strap positioning control to position the arm and shoulder in the desired position. It allows placement of the brace by the patient with a single hand. The design facilitates the patient bracing maximum functionality and freedom of movement of the arm. Made with a breathable soft. S (23 cm - 28 cm.); M (28 cm - 34 cm). L (34 cm - 40 cm). (Biceps contour)

LIF600 forearm brachial Lifter Orthotic device comprising a support shoulder pad comprising a forearm support and joined by a set of straps with Velcro micro-adjustment means. The pad includes a core of rigid plastic to improve the function of the weight of the braço. La traction / suspension arm support is provided by two straps. Additionally it has a control rod positioning to place the arm and shoulder in the desired

it to the shoulder and elbow pad / forearm support. The garments should be used strictly as directed by the physician or technician. If an allergic reaction should occur remove the product and consult with the prescribing physician. Positioned such that compression is not excessive. In case of doubt in choosing the size choose the larger size. Used only during periods of activity. Do not expose the garment to a flame or heat source at very high temperatures. Store at room temperature, preferably in the original packaging. Guarantee within the framework of laws.

Precautions

Follow the manufacturer's directions for proper orthosis placement. Follow the instructions or recommendations of the prescriber or healthcare personnel. For a personalized adaptation, consult a specialist. If there is ongoing pain, it is advisable to contact a healthcare professional. If an allergic reaction should occur remove the product and consult with the prescribing physician.

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