
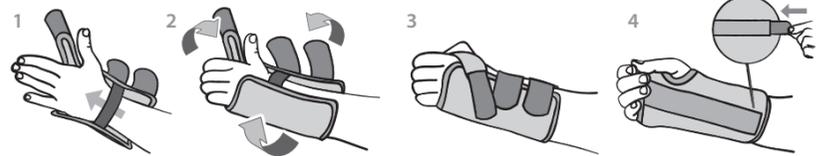
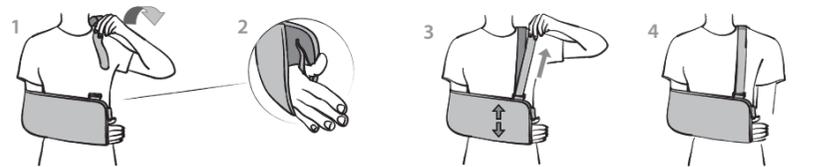
## KMQ401


## KSP401
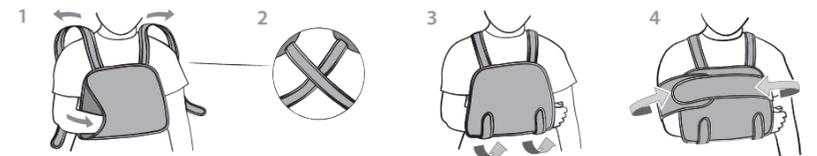
## KSH401

## KCL401

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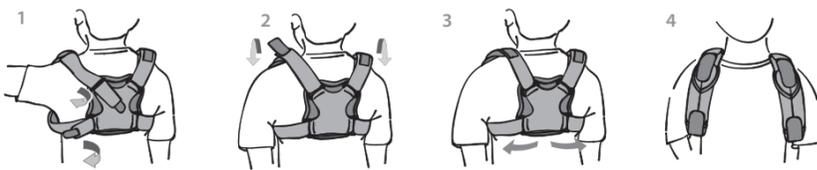Orthopedic Medical Specialties s.l. guarantees all its products as long as they have not been tampered with or altered and always within the framework and deadlines of the legal provisions. Products whose characteristics have been altered due to misuse are not included in this warranty.

#### Normative

The medical device complies with Regulation (EU) 2017 /745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

#### Description and characteristics

KM401 Wrist immobilizer: Velcro closure. It incorporates a formed and removable malleable aluminum splint. KSP401 Bag support: Extremely breathable fabric. With loop to facilitate the rest of the thumb and the position of the wrist. Protective shoulder pad. KSH401 Shoulder immobilizer: Gleno-umeral joint immobilizer support. Distributes the weight of the treated arm avoiding overload of the cervical area. Straps crossed over the back to improve support. KCL401 Clavicle immobilizer: inelastic straps in "eight" or "eight" of guarismo. Traction regulation by four independent adjustment points. Large pads that adapt easily and prevent chafing and overpressure. KR401 Knee immobilizer: made of comfortable and breathable material. It incorporates three lateral and rear conformable wheels. They can be adapted to achieve the desired degree of flexion. Release the kneecap. Perfect adaptation to all the perimeters of the leg thanks to its two mobile side panels. Velcro closure. KR461 Enveloping knee brace: Made with breathable and elastic material. Hosing for the kneecap, avoiding pressure on it and facilitating flexo-extension. Side spiral rods Elastic handles that regulate compression on the thigh and calf. Popliteal area open. Easy placement, control and review of the area. KT461 Stabilizer ankle brace: Elastic and breathable fabric. Open on the calf for easy placement. It incorporates an elastic strap that allows the realization of a stabilizing bandage. KPC401 Cranial protector: Made with padded and breathable material. Padded chin guard. KCC401 Cervical collar: soft padding. Designed to easily adapt to the morphology of the patient. Adjustment and placement by velcro.

#### Efectos

KM401 Wrist immobilizer: Fastening. Stabilization of the wrist joint. Controlled immobilization KSP401 Bag support: supports the weight of the arm. Distribute the weight of the arm over the entire orthosis, including the elbow. KSH401 Shoulder immobilizer: immobilizes the shoulder joint in internal rotation. Controls flexo-extension, rotation, abduction and shoulder adduction. It keeps the elbow in flexion of 90° and in intermediate prono-supination. KCL401 Clavicle Immobilizer. It facilitates the formation of the fracture callus approaching the segments by retroupsion of the shoulders. It exerts a wide traction on the acromio-clavicular and coraco-clavicular joint. KR401 Knee immobilizer: Immobilization and protection of the knee joint. KR461 Enveloping knee brace: Stabilization. Stimulation and improvement of proprioception. KT461 Stabilizer ankle brace: adjustable damping of the joint by elastic strap. Compression. Improvement of proprioception. Therapeutic heat KPC401 Cranial protector: Cranial protection. KCC401 Cervical collar: reminder to restrict head and neck movements. Support and support. It reduces the pressure on the intervertebral discs. Relief of muscle tension.

#### Indications

KM401 Wrist immobilizer: Para-articular wrist trauma without fracture. Recurrent wrist sprain. Sequela painful or inflammatory. KSP401 Bag support: Shoulder and arm injuries. Subluxation Soft tissue injuries (burns, infections). Inflammatory or painful sequelae Postoperative. KSH401 Shoulder immobilizer: Subcapital humerus fractures. Clavicle fracture. Subluxation of the Gleno-umeral joint. Acromio-clavicular dislocation. Shoulder and arm trauma treatment. Post surgery KCL401 Clavicle immobilizer: Clavicle fracture. Postoperative. KR401 Knee immobilizer: Trauma. Conservative treatment of kneecap fractures. Sprain of ligaments. Pre-surgical and post-surgical immobilization. KR461 Enveloping knee brace: Knee sprain. Tendinitis Sprain. KT461 Stabilizer ankle brace: Sprain, Tendinitis, Sprain relapse prevention: KPC401 Cranial protector: Cerebral palsy, Epileptic patients KCC401 Cervical collar: Trauma. Cervicobrachialgia. Postoperative protection.

#### Instructions for use and placement

KM401: Adapt the strap to the morphology of the patient. Place the orthosis over the area at the correct height for use. Use velcro straps to close and adjust. KSP40: insert the forearm completely into the support. The elbow should reach the apex of the bag and the wrist joint should be inside the support. Pass the webbing behind the neck and adjust. The webbing can also be passed between the arm and the trunk and then over the contra-lateral shoulder. Then adjust. KSH401: insert the arm into the support. Pass the straps over the shoulders, cross them on the back and fix them in the front area of the support. The additional band is then adjusted by fixing it on the area that hugs the forearm, passing it around the trunk and fixing its other end on the area of the orthosis that hugs the forearm. KCL401: Place the two axillary padded supports on the arm. Pass a cinch from the end of the axillary pad over the shoulder to the back of the dorsal pad and fix with velcro. Pass the other webbing from the end of the axillary pad below the armpit to the dorsal pad pin and fix with velcro. Repeat the process on the other arm. Traction of each of the four adjustments symmetrically until the shoulders are retracted.

The orthosis should be completely centered between the patient's scapulae and should be pulled with the same force on both shoulders. KR401: Adapt the aluminum bars to the morphology of the patient. Remove the side panels. Then place the patient's leg on the central panel at the correct height (align the kneecap at the level of the patellar orifice of the garment). Subsequently, fix the side panels according to the perimeter measurement of the patient. Finally close and adjust using the straps of Velcro KR461: The patellar orifice should be well centered on the kneecap. Embrace the leg with the adjustment bands and fix them using velcro. pronator for bandage in "eight" or stabilizer. KPC401: insert the head into the helmet and check that the head is well oriented. Adjust the chin strap so that the helmet is placed securely and stably. KCC401: place around the neck Checking that the chin notch is well centered.

#### Precautions

Follow the manufacturer's directions for proper orthosis placement. Follow the instructions or recommendations of the prescriber or healthcare personnel. For a personalized adaptation, consult a specialist. If there is ongoing pain, it is advisable to contact a healthcare professional. The used material is hypoallergenic but if there is any allergic reaction or irritation remove the product and consult with the prescribing doctor. The user should position itself in such a way that the compression is not excessive. In case of doubt about the size, choose the bigger size. Only use during active periods. Remove during the night. The correct choice of the appropriate size for each patient is essential to obtain the highest degree of therapeutic efficacy. The adjustments of the orthotic should be made individually. The orthotic should be used for the purpose it is prescribed for. Do not expose the orthotic to a flame, heat source or high temperatures.

#### Washing and maintenance

Hand wash with warm water (maximum 30°) and neutral soap. During washing, if the garment has velcro, they must be glued together or kept closed. Dry at room temperature. Do not store or iron the garment. Contact with ointments, creams, alcohols and solvents can damage the garment. Do not use bleach or dry clean. To dry, spread the garment and do not expose it directly to a direct heat source. Store the orthosis at room temperature, preferably in the original container.

#### Quality Certificates

All the materials used have been tested, comply with the applicable European regulations, are of high quality and offer maximum safety and comfort during use. Oko-Tex Standard 100 Certificate: Ecological textile label granted after the creation of a catalog of analysis criteria that guarantees the absence of substances harmful to health. It certifies the absence of harmful substances in all the materials with which a garment is made.

#### Warranty

Especialidades Médico Ortopédicas S.L. guarantees all its products as long as they have not been tampered with or altered and always within the framework and deadlines of the legal provisions. Products whose characteristics have been altered due to misuse are not included in this warranty.

#### Notice

To dispose of the product, use the solid waste container. For the disposal of the container, depending on its material, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of the country. Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

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