Epigrip

Dear Customer, We appreciate the confidence placed on a product EMO. Choosing a brace Epigrip brand you have purchased a quality product and high level medical doctor. Read these instructions carefully. If you have any questions please contact your doctor or your specialist.

REGULATIONS

The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

DESCRIPTION AND FEATURES

CD100 Epigrip epicondylitis Band: Thanks to its anatomical shape prevents and limits the progression of the lesion to protect the muscle fascia. Adaptive features an elastic component that facilitates adaptation to the morphology of the forearm performing a dynamic compression. Effective: soul of rigid material which distributes the pressure evenly.

Technologically Advanced: manufacture by microfusion avoids overpressure and abrasions. Light: made with materials for maximum strength and minimum thickness. Adjustable via inelastic textile handle. incorporates a removable cushion made epicondylar visco-elastic material, non-slip. Plus Epigrip

CD300 epicondylitis Band: Thanks to its anatomical shape prevents and limits the progression of the lesion to protect the muscle fascia. Adaptive features an elastic component that facilitates adaptation to the morphology of the forearm performing a dynamic compression. Effective: soul of rigid material which distributes the pressure evenly. Technologically Advanced: manufacture by micro-fusion avoids overpressure and abrasions. Light: made with materials for maximum strength and minimum thickness. Adjustable via inelastic textile handle. incorporates a removable cushion made epicondylar visco-elastic material, non-slip. It incorporates a flexible piece of plastic shaped anatomically that distributes pressure evenly over the forearm.

EFFECTS

The role of counter-force to relieve overload produced on epicondileos muscle attachments, performing a function ancillary or supplementary insertion.

INDICATIONS

Inflammation of the fascia muscle, epicondylitis ("tennis elbow"), epicondylitis ("golfer's elbow").

INSTRUCTIONS FOR USE AND PLACEMENT

The product should be adjusted by the prosthetist or the physician individually for each patient. The correct choice of size is essential for maximum effectiveness of treatment and prolong the life of the garment.

The user must put the brace so that the strap that's built to be about three inches away from the angle of flexion of the elbow joint. The fit of the brace should be done with relaxed muscles so that when you make a muscle tension strap protects the muscle fascia.

PRECAUTIONS

Follow the manufacturer's directions for proper orthosis placement. Follow the instructions or recommendations of the prescriber or healthcare personnel. For a personalized adaptation, consult a specialist. If there is ongoing pain, it is advisable to contact a healthcare professional. If you experience any allergic reaction to remove the product and consult with the prescribing physician. For small nuisance caused by excessive sweating recommend using a cotton shirt. Positioned so that compression is not excessive. Do not expose the garment to a flame or heat source at high temperatures. Store at room temperature, preferably in the original packaging. Guarantee within the framework of laws.

CLEANING AND MAINTENANCE

Hand wash with warm water (max. 30 °) and neutral soap. During washing, if the garment has Velcro, they must stick together or keep them closed. Dry at room temperature. Do not iron the garment shop. Contact with ointments, creams, alcohol and solvents can damage the garment.

QUALITY CERTIFICATES

All materials used have been tested, comply with applicable European standards, are high quality and offer maximum safety and comfort during use. Öko-Tex Standard 100: textile eco-label awarded after the completion of a catalog of criteria for analysis to ensure the absence of substances harmful to health. Certify the absence of harmful substances in all the materials that is made of a garment.

NOTICE

To dispose of the product, use the solid waste container. For the disposal of the container, depending on its material, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of the country. Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

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