Lumbofit

en

Dear customer, we thank you for the trust placed in an EMO product. By choosing an orthosis from the Lumbofit line you have acquired a high quality medical product. Read these instructions carefully. If you have any questions, please contact your doctor or your specialist establishment.

INDICATIONS


LS601, LS602 and LS605: Acute and subacute low back pain, Lumboxicity, Facet Syndrome, Discal protrusion, Herniated disc, Spondylosis, Spondylarthritis, Degenerative Discopathies, Degenerative processes, Muscle hypotonia, Postural rehabilitation. Postoperative LS608 and DLS609: Low back pain and back pain, Lumboxicity, Facet Syndrome, Discal protrusion, Herniated disc, Spondylosis, Spondylarthritis, Degenerative Discopathies, Degenerative processes, Muscle hypotonia, Postural rehabilitation. Postoperative.

DESCRIPTION AND CHARACTERISTICS

LS601, LS602, LS605, DLS608 and DLS609: place the strip on the sacrolumbar zone at the correct height for use. Help the fingers to tighten and tighten the garment over the abdomen with loose sidebands. Close the garment with velcro once obtained the desired compression. Then adjust the garment by pulling first the hip level handles and then pulling the handles of the waist level. With what it is possible to discharge the lumbar zone thanks to the lateral corrugados that there are to both sides of the orthosis. Finally regulate the discharge and lumbar support with the help of the lateral bands and fix them on the abdominal area. DLS608 and DLS609: pass the straps above the shoulders, then under the armpits, cross over the dorsal area, surround the thorax and fix them with velcro on the abdominal phona.

WASHING AND MAINTENANCE

Hand wash with warm water (maximum 30°) and neutral soap. During washing, if the garment has Velcro, it must be glued together or kept closed. Dry at room temperature. Do not store or iron the garment. Contact with ointments, creams, alcohols and solvents may deteriorate the garment.

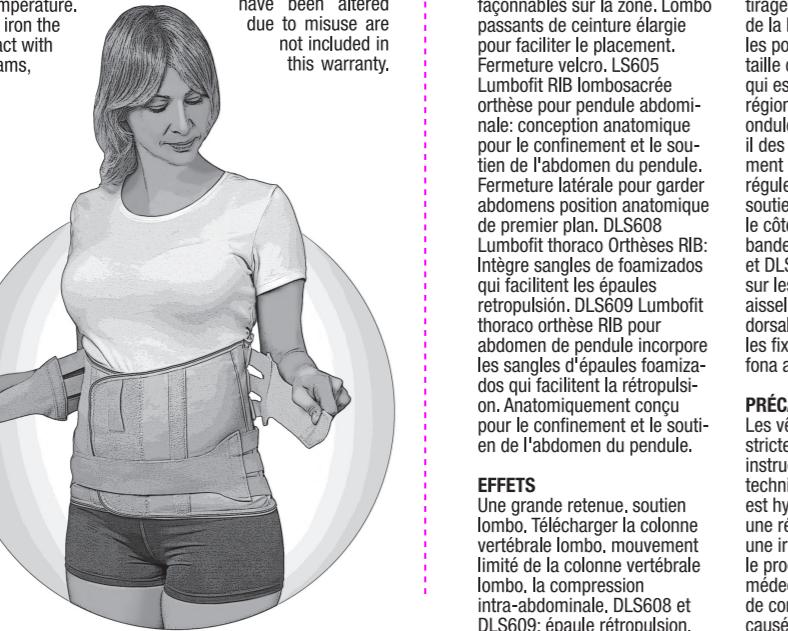EFFECTS

Great containment. Lumbosacral support. Download of the lumbosacral spine, Limitation of the mobility of the sacrolumbar rachis. Intra-abdominal compression. DLS608 and DLS609: Shoulder Retropulsion.

INSTRUCTIONS FOR USE

LS601, LS602, LS605, DLS608 and DLS609: place the strip on the sacrolumbar zone at the correct height for use. Help the fingers to tighten and tighten the garment over the abdomen with loose sidebands. Close the garment with velcro once obtained the desired compression. Then adjust the garment by pulling first the hip level handles and then pulling the handles of the waist level. With what it is possible to discharge the lumbar zone thanks to the lateral corrugados that there are to both sides of the orthosis. Finally regulate the discharge and lumbar support with the help of the lateral bands and fix them on the abdominal area. DLS608 and DLS609: pass the straps above the shoulders, then under the armpits, cross over the dorsal area, surround the thorax and fix them with velcro on the abdominal phona.

WARRANTY AND MAINTENANCE

Specialties Orthopedic Physician s.l. Guarantees all its products as long as they have not been manipulated or altered and always within the framework and the terms of the legal provisions. Products whose characteristics have been altered due to misuse are not included in this warranty.

PRECAUTIONS

The garments must be used strictly according to the instructions of the doctor or specialized technician. The material used is hypoallergenic but if there is any allergic reaction or irritation remove the product and consult with the prescribing physician. In case of minor discomfort caused by excessive sweating we recommend the use of a

cotton t-shirt. Position in such a way that the compression is not excessive. In case of doubt in the choice of size choose the larger size. Use only during periods of activity. Reduce abdominal pressure during day time breaks or after meals. Remove during night time rest. The correct choice of the appropriate size for each patient is essential to obtain the highest degree of therapeutic efficacy and prolong the useful life of the orthosis. The adjustment and adaptation of the orthosis must be individual and performed by a specialist or specialized technician. The use of this orthosis is conditioned to the indications of the prescriber so it must be used for the purposes indicated by him. Adapt the steels to the shape of the patient's spine. Do not expose the garment to a flame or heat source at very high temperatures.

WASHING AND MAINTENANCE
Hand wash with warm water (maximum 30°) and neutral soap. During washing, if the garment has Velcro, it must be glued together or kept closed. Dry at room temperature. Do not store or iron the garment. Contact with ointments, creams, alcohols and solvents may deteriorate the garment.

NORMATIVE

The products are in accordance with the European Directive on Sanitary Products 93/42 / EEC (Royal Decree 1591/2009). There has been performed a risk analysis on these products to minimize all possible risks prEN 1441 (UNE EN ISO 14971)

QUALITY CERTIFICATES

All the materials used have been tested, comply with the applicable European regulations, are of high quality and offer maximum safety and comfort during use. Oko-Tex Standard 100 Certified: Textile eco-label awarded following the completion of a catalog of test criteria ensuring the absence of substances harmful to health. Certifies the absence of harmful substances in all materials with which a garment is made.

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