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BEFORE USE

We do not recommend that the product is issued to anyone without carrying out a full assessment of their specific need first. It is important that the user is given the necessary support by the product as prescribed by a qualified professional. This is both for the safety of the user and the rehabilitation process.

INTENDED PURPOSE

A walking aid to assist those who have a limited walking ability.

INTENDED USERS

Any mobility restricted user who is up to 136kg in weight

SAFETY INSTRUCTIONS

1. Consult a healthcare professional before using any assisted mobility device.
2. Not intended for use on ice or other slippery surfaces as it has the potential to slip.
3. Ensure the push buttons have securely locked in position.
4. Exercise caution when walking on sloping / slippery ground and on uneven surfaces.
5. Be careful when walking backwards over obstacles.
6. Never sit and push with your feet.
7. Do not go up or down stairs with the walker.
8. Do not use the walker on escalators.
9. Do not under any circumstances stand on the walker.
10. Be careful when using long skirts, the clothes might get stuck in the wheels.

ASSEMBLY

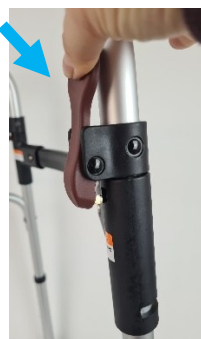
1. Remove the walker and all components from the packaging
2. Insert the legs into the tube (no tools required), just push.

3. Insert the bar to secure both sides of the walker.

4. To close the walker, simply press the tab. To open it, simply pull the side and let the tab engage correctly until it locks in place.

CLEANING

Use only a damp cloth to remove dust, use a damp cloth with neutral soap to remove stubborn dirt. Dry thoroughly before use. Use water at a temperature no higher than 30 °C. Do not use water and/or steam melting machines.

MAINTAINENCE

The walker should be regularly inspected for signs of damage or wear and tear. Please check for:

- Damage to tubing, including any bends or cracks at the tube joints.
- Worn or loose push buttons or elongated holes
- Loose or worn handgrips

ACCESORIES

1. 360º Rotation Walker Wheel
2. Wheel with shock absorbers

WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

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