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## BEFORE USE

We do not recommend that the product be given to anyone without first carrying out a full assessment of their specific need. It is important that the user receives the necessary product support as prescribed by a qualified professional. This is both for the safety of the user and for the rehabilitation process.

## INTENDED PURPOSE

The rollator is intended to be used for people who are experiencing generalized weakness, difficulty with balance, a lower extremity injury or surgery that need additional support to maintain balance and stability while walking outdoors or used for rehabilitation.

## DEVICE DESCRIPTION

The rollator consists of support platform (platform support or forearm support), handle, handle rod, handle rod adjustment, wheels, height adjustment, (parking) brake. The device is composed of a folding mechanism and a seat. The main frame material is high-strength aluminum pipe or steel pipe.

## SAFETY INSTRUCTIONS

1. Check whether the handle installation is loose or burrs, so as not to affect the safe use.
2. Check whether the wheels are on the ground smoothly and whether the fittings are stable.
3. The rollator is an auxiliary tool for walking, please do not use it as an overweight support item other than auxiliary tools.
4. The user's weight shall not exceed 136KG.

5. Rollators are not suitable for mountaineering.

## USAGE METHOD

1. The handle must be installed before using rollator: unscrew the knob of the fixed handle, insert the handle into the inner tube of the bracket by left and right and adjust it to the appropriate height for the user, then use the five-star knob to lock it. The frame needs to be fully opened until all the wheels are stable and can be used.
2. Temporarily brake when in use: Lift the brake lever forcefully to brake. Long-term parking: Hold the brake lever with your hand and press it down firmly. When you hear a "click", the parking brake opens.
3. Fold up the seat board, then take out the shopping basket, and lift the round steel under the shopping basket to fold it.

## MAINTAINENCE

Use a soft dry cloth when wiping the rollator. Keep the rollator clean and tidy.

## STORAGE & TRANSPORTATION CONDITION

Should be stored in clean, ventilated and dry place.  
Rainproof, moisture-proof, and handle with care during transportation.

## DISPOSAL

Please dispose the product after use to comply with local regulation.

## WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

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