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BEFORE USE

We do not recommend that the product is issued to anyone without carrying out a full assessment of their specific need first. It is important that the user is given the necessary support by the product as prescribed by a qualified

professional. This is both for the safety of the user and the rehabilitation process.

Toilet and shower chair BW600 made of aluminium. Easy to clean and height adjustable seat. Three functions in one chair; toilet chair with built-in urinal bowl as standard, sturdy shower chair and toilet lift with armrests and backrest. The front-opening U-shaped seat makes it easy to wash and take care of yourself. Seat cover and backrest made of padded material.

SAFETY INSTRUCTIONS

1. Ensure that all the push buttons lock into position and go through the holes in the legs.
2. Check that all the fastenings are correctly tightened.
3. Check the rubber tips for rips, wear & tear, replace if necessary.
4. Never stand on the seat.
5. Always sit in the centre of the bench, do not lean on the edges of the bench/chair as this can cause instability.
6. Do not exceed the maximum weight limit of 136kgs.
7. The bench/chair is only suitable for use by one user at a time.
8. Don't attempt to lift a commode by the armrests.
9. Warning! Keep cigarettes and sources of heat and flames away from this product.

MAINTAINENCE

The shower chair should be regularly inspected for signs of damage or wear and tear. Please check for:

- Damage to tubing, including any bends or cracks at the tube joints.
- Worn or loose push buttons or elongated holes
- Loose or worn handgrips

USEFUL LIFE

5 years.

WASTE DISPOSAL

Please dispose of the product after use to comply with local regulations.

WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

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