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**BEFORE USE**

We do not recommend that the product is issued to anyone without carrying out a full assessment of their specific need first. It is important that the user is given the necessary support by the product as prescribed by a qualified professional. This is both for the safety of the user and the rehabilitation process.

## SAFETY INSTRUCTIONS

1. Ensure that all the push buttons lock into position and go through the holes in the legs.
2. Check that all the fastenings are correctly tightened.
3. Check the rubber tips for rips, wear & tear, replace if necessary.
4. Never stand on the seat.
5. Always sit in the center of the bench, do not lean on the edges of the bench/chair as this can cause instability.
6. Do not exceed the maximum weight limit of 136kgs.
7. The bench/chair is only suitable for use by one user at a time.

## MAINTAINENCE

The chair should be regularly inspected for signs of damage or wear and tear. Please check for:

- Damage to tubing, including any bends or cracks at the tube joints.
- Worn or loose push buttons or elongated holes
- Loose or worn handgrips



## WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

## ASSEMBLY INSTRUCTIONS

Accessories:

1. Fixing supports – 2 pcs.
2. Armrests – 2 pcs.
3. Seat – 1 pc.
4. Seat base – 1 pc.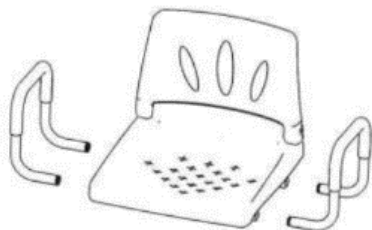
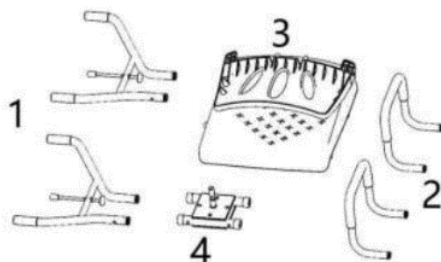


**Step 1:** Raise the seat back and insert the armrests into the counters holes on the seat.


Note: Armrest buttons should come out of the holes in the seat.

**Step 2:** Install the fixing supports in holes on the base of the seat.

Note: When installing the button to the supports should come out of the holes.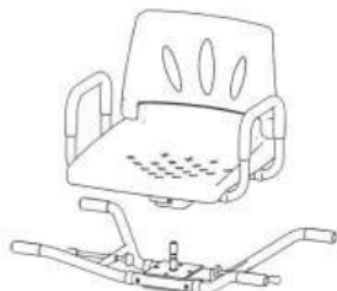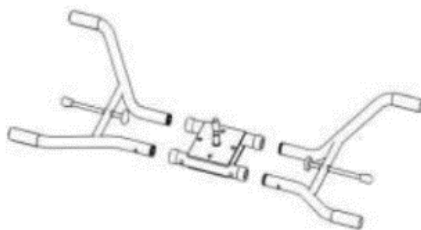

**Step 3:** Carefully place the seat on base rod, tighten the handle on the right so that the seat and base are not separated.

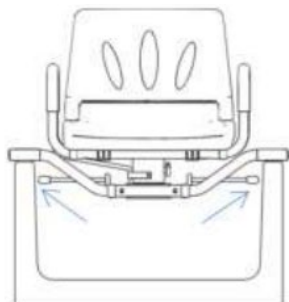**Step 4:** Install the seat on the bathtub.

Secure the seat to the bathtub with using bolts on both sides of seats.

**Step 5:** Lift the lever up to turn the seat and release the lever to stop turning.

Push the seat slightly left and right to the bottom of the lever entered the hole on the base.

**Note:**

To make sure that armrests and handrail brackets properly secured locking buttons on the armrests and handrail brackets must be completely come out of the holes in the seat and base.

After turning the seat, make sure that the lever fixed at the base.

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