**Bastón Aluminio Extensible Silver**  
Manual de instrucciones

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## BEFORE USE

We do not recommend that the product is issued to anyone without carrying out a full assessment of their specific need first. It is important that the user is given the necessary support by the product as prescribed by a qualified professional. This is both for the safety of the user and the rehabilitation process.

## INTENDED PURPOSE

A walking aid to assist those who have a limited walking ability.

## INTENDED USERS

Any mobility restricted user who is up to 136 kg in weight

## SAFETY INSTRUCTIONS

1. Consult a healthcare professional before using any assisted mobility device.
2. Not intended for use on ice or other slippery surfaces as it has the potential to slip.
3. Ensure the push buttons have securely locked the walking stick sections in position.
4. If the handle or tip becomes loose, or if the walking stick becomes damaged, stop using the walking stick immediately.

## MAINTAINENCE

The walking stick should be regularly inspected for signs of damage or wear and tear. Please check for:

- Damage to tubing, including any bends or cracks at the tube joints.
- Worn or loose push buttons or elongated holes
- Loose or worn handgrips

## WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

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