Mesa auxiliar de cama plegable PREMIUM
Manual de instrucciones

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This manual **MUST** be given to the user of the product **BEFORE** using this product, read this manual and save for future reference.

The Overbed Table is a mobile table with an adjustable height surface designed to be used with various types of adjustable beds and chairs. This table can meet both patient and caregiver needs by providing a surface for eating and working, along with personal storage space.

This product is designed for indoor use only. It should not be used near open flames and heat.

Maximum weight limitation for this product is 15kgs;

DO NOT overload the product as that could result in breakage, as well as damage to property and/or serious injury.

Use **ONLY** as for its intended use this manual mentioned. This product should not be used: as a sleep top; as a mobility aid to assist a person in walking; sitting down or standing up; or to transfer anything.

WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

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