FOLDING WALKING STICK BM115

Purpose:

A walking aid to assist those who have a limited walking ability.

Features:

Aluminium, PVC

Handgrip: TPR

Maximum weight: Users up to 150 kg.

1 Non-slip tip

Size: 80 to 93 cm

Batch: See package

Color: Black

Date of manufacture: See package

Model: BM115

Brand: BMotion

Maintenance and storage:

Clean method: use wet cloth to wipe .

Clean the grips with a mild soap, detergent or household cleaner, do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen. The tips on this product contain dry natural rubber.

1.Periodically inspect the cane for damage or loose components.

2.Check tips are in good condition. To preserve the life of the tips. Replace tips immediately when worn.

Instructions for use:

1. When using a cane for stability, you may grip it in either hand.
2. Hold the cane in the opposite hand of the affected leg and move the cane unison with the affected leg. Each time you step with the affected leg, also move the cane to give you support as you walk. When you step forward with the unaffected leg, keep the cane in place for support.
3. Your healthcare provider might recommend a specific walking pattern.
4. Push button adjustable.
 - 4.1 Loosen the locking collar.
 - 4.2 With your shoes on, and your arm down at your side, your wrist should be even with the handle of the cane. Engage the spring button into the locking collar.
 - 4.3 Retighten the locking collar.
5. Quad canes
 - 5.1 Adjusting for right- or left-hand use
 - 5.2 Loosen the locking collar
 - 5.3 Depress the spring bottom
 - 5.4 Rotate the handle ½ turn engaging the spring button in a hole that provides correct height.
 - 5.5 Retighten the locking collar.

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