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DESCRIPTION

Bath Chair is used as bathing tools for people with disabilities, patients and the elderly who are difficult to maintain balance while showering. The Bath Chair is made of PE, PU, Aluminium alloy and other materials. There are several models of Bath Chair which can meet different requirements.

Intended Use

Bath Chair is used as bathing tools for people with disabilities, patients and the elderly who are difficult to maintain balance while showering.

INSTALLATION INSTRUCTIONS

1. Before use, adjust the bath seat to a suitable height, so that four feet are on the ground smoothly.
2. Check whether the backrest is installed firmly and whether there is a swing during use. Please sit in the middle of the shower chair during use.
3. The height of the bath seat can be adjusted. When adjusting the height, first push down the marbles, and then pull it down. Just adjust to the height suitable for your own use.

MAINTENANCE

1. Routinely clean and disinfect once a day by medical staff or users, and thoroughly disinfect the chair when changing patients.
2. Do not contact with strong acids and alkalis to prevent corrosion.

WARNINGS

- Please do not support on one side when getting up or sitting down, to prevent the shower chair from being overturned and injured.
- Use it with care on even ground. Please clean and disinfect the bath chair in time.
- This product is safe to use with a load of 136kg. Be careful when using it in excess of this weight, or it can be customized by the company.

STORAGE

The product should be stored in normal temperature and humidity and keep dry.

SHELF LIFE

5 years.

WASTE DISPOSAL

Please dispose of the product after use to comply with local regulations.

WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

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