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## DESCRIPTION

Comfortable chair with built-in toilet, its discreet appearance conceals the function for which it has been manufactured. Designed for people who for some reason have difficulty getting to the toilet. Easy to clean and height adjustable seat.

Coated steel structure. Wooden seat and backrest with upholstery. PVC upholstery and ABS bedpan.

## Partes

- 1- Backrest
- 2- Mobile seat
- 3- Toilet seat
- 4- Armrests
- 5- Pan with lid
- 6- Adjustable legs
- 7- Fixing clips

## INSTALLATION INSTRUCTIONS

1. Adjust the toilet seat to the required height.
2. Lift the seat cover.
3. After use, close the toilet pan lid and lift it straight off the toilet pan support.

## MAINTENANCE

1. Routinely clean and disinfect once a day by medical staff or users, and thoroughly disinfect the chair when changing patients.
2. Do not contact with strong acids and alkalis to prevent corrosion.

## WARNINGS

- 1- Do not use if you detect that any of the parts of the wheelchair are in poor condition.
- 2- Be sure to adjust all four legs to the same height and check that the chair is on a flat, stable surface.
- 3- Do not exceed the maximum weight indicated in the manual and on the product label, which is 130 kg.
- 4- Make sure that the product is adapted to the characteristics and physical condition of the user.
- 5- Do not stand on the seat. It is not suitable for use as a climbing aid.

## STORAGE

The product should be stored in normal temperature and humidity and keep dry.

## SHELF LIFE

5 years.

## WASTE DISPOSAL

Please dispose of the product after use to comply with local regulations.

## WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

## ASSEMBLY

- 1- Backrest: Insert the tubes in the indicated place and fix them with the clips as shown in the picture.

- 2- Legs: Insert the legs into the chair supports. Secure them with the clip as shown in the picture.

- 3- Seat: Removable by pressing lightly on the hooks on the tube.
- 4- Urinal: It goes in the guide under the seat.

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