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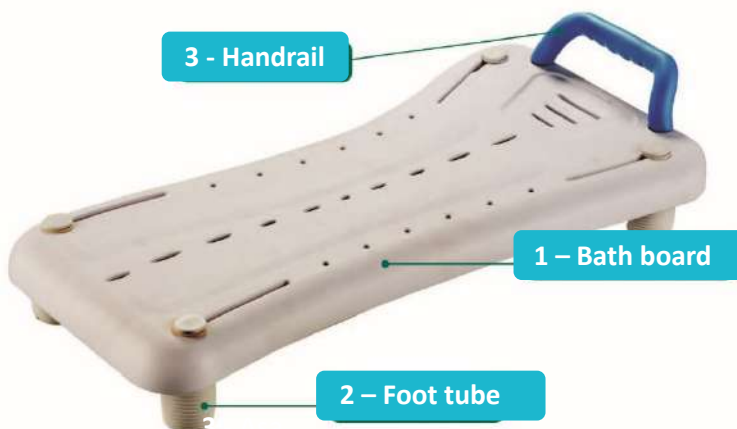
ATTENTION



1. The maximum weight of this product is: 160kg Before use, check whether all parts of the shower board are firm and ensure that there are no loose defects.
2. Before use, check whether all parts of the shower board are firm and ensure that there are no loose defects
3. Do not tip your body too far forward to prevent falling.
4. The packaged products should be stored in a room with a relative humidity of no more than 80%, dry, wind-exposed and non-corrosive substances.
5. Wipe the product with a soft dry cloth.
6. The waste shall be disposed of in accordance with the city's regulations on environmental protection.
7. Product life: 3 years (except consumable parts)

INSTALLATION AND USE INSTRUCTIONS

INSTALLATION AND USE INSTRUCTIONS

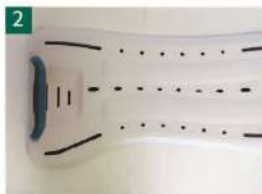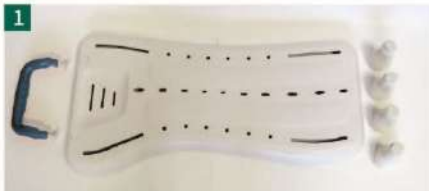1. Carefully take out the product from the transport carton, and the product components refer to Figure 1:1 shower plate, 1 handrail with plastic screws, and 4 foot tube sets;

2. Align the handrail with the hole and fasten the plastic screws at the bottom, as shown in Figure 2 and Figure 3;

3 As shown in Figure 5, lock the plastic screw with the foot tube, lock the four foot tube, shake to confirm whether the installation is successful

4. Move the plastic screw with adjustable width as shown in the figure

WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

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