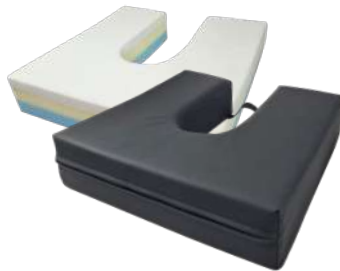
+ Confort

ALMOFADAS ANTI ESCARAS – MANUAL DE UTILIZAÇÃO
COJINES ANTIESCARAS – MANUAL DE USUARIO
ANTIDECUBITUS CUSHION – USER MANUAL

COUSSINS ANTI-ESCARRES – MANUEL UTILIZATEUR
CUSCINI ANTI-ULCERA – MANUALE UTENTE
ANTI-DEKUBITUS KISEN – BENUTZERHANDBUCH

Cojines Antiescaras:

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Antidecubitus Cushion:

The antidecubitus cushion prevent the appearance of pressure ulcers, consequence from the permanence of the body in the same position for long periods of time. Pressure ulcers appear mainly in areas of bony prominence and support points of the body, such as heels, gluteus, shoulders, knees, elbows and pelvis. Excessive pressure on a point of the body, limits the intake of blood to that area and consequently oxygen, causing redness, destruction of the dermis/epidermis and consequently superficial or deep wounds.

To avoid the appearance of pressure ulcers, caregivers should use different solutions: Regularly change the positioning of the body; Use cushions or devices that minimize pressure at each point of support; Use moisturizing skin creams.

Model Selection and Usage Indications:

According to the risk of Pressure ulcers (Braden scale or Norton) and weight of the user.

EvoluXion seating cushions are suitable for both wheelchairs and conventional seats, such as sofas, chairs, seats, etc.

Classification:

EvoluXion seating cushions are considered Class I medical devices, in accordance with Directive 2017/745/EU.

Evolution Health, Lda
Rua Hermano Neves, 18 – 3º Piso – E7
1600-477 Lisboa
Portugal
export@evolutionhealth.com
www.evolutionhealth.com

Model	Dimensions	REF.	Max weight user	Indications and composition	Degree of Risk*
+Confort Square	42X42X8 cm	EV100	90 kg	Confort and Wound Prevention. Three layers: 2cm 60Kg/m3 density Memory Foam + 3cm 33Kg/m3 density HR foam+ 3cm 35Kg/m3 density HR foam. PU fabric cover.	Medium/Low
+Confort HorseShoe Shape	42X42X8 cm	EV105	90kg	Confort and Wound Prevention. Three layers: 2cm 60Kg/m3 density Memory Foam + 3cm 33Kg/m3 density HR foam+3cm 35Kg/m3 density HR foam, PU fabric cover.	Medium/Low
+Confort Round	∅42X8 cm	EV110	90kg	Confort and Wound Prevention. Three layers: 2cm 60Kg/m3 density Memory Foam + 3cm 33Kg/m3 density HR foam+3cm 35Kg/m3 density HR foam, PU fabric cover.	Medium/Low

Cleaning and Maintenance:

Cover:

All cushions bring a removable cover, which allows you to wash. It can be washed by hand with cold water, using non-abrasive products or can be washed in the machine, using a program for delicate, cold and centrifuge-free clothing. It is recommended to use a safety net to avoid deformation of the cover during washing.

Dry in the Horizontal position.

Do not use bleach.

Don't iron.

Cushion Core: Viscoelastic foam block - Do not wet.

* - Braden or Norton Tables 

Safety:

Make sure that the cushion fits the seat properly.

Storage:

Store the cushion preferably horizontally, avoiding sun exposure and moisture.

Precautions:

- Avoid contact of the cushion with solvents, corrosive products and sharp objects.
- Regularly check the normal wear and tear of the cushion, such as appearance modifications, possible snares and/or holes.
- Avoid prolonged contact with liquids.
- Avoid sun exposure;
- Avoid moisture;
- Regularly carry out a visual control of the condition of the cushion, as the aging and wear of the cushion compromises the properties of the cushion, and consequently its function of preventing pressure ulcers.

Warranty:

Period of 3 years from the date of purchase. The warranty does not cover the normal wear and tear of the product, as well as inappropriate use of the product. For complaint, contact the point of sale where the cushion was purchased, being mandatory, the presentation of proof of purchase with date and the information of the Article Lot.

Symbols and Legend:

	Attention. Read the Instructions first
	Manufacturer
LOTE	Lot
REF	Reference within the EvoluXion Range
	Read the Instructions first.
CE	Certified in accordance with Directive 2017/745/EU

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