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DESCRIPTION

Assistive device for washing and showering. It is a technical aid that allows people with reduced mobility or disability, who cannot make conventional use of the shower, to remain seated during bathing. It is also an option to improve the quality of life of people suffering from urinary incontinence.

GRP (glass-fibre reinforced nylon) structure. Can be used underwater. 12 cm PVC wheels. Can be used directly in the toilet. Includes bucket that can be inserted at the back. Folding armrests and removable footrests.

INSTALLATION INSTRUCTIONS

1. Before using the wheelchair, it is advisable to learn the techniques for getting up, sitting down, going up and down steps, ramps, etc.
2. Do not go up and down stairs without assistance.

3. It is not advisable to lean forward (to tie your shoes, pick up something from the floor, etc.), the user could fall.
4. Never drive the wheelchair on the rear wheels.
5. Never transport the wheelchair by the armrests, as this can damage the wheelchair structure. Make sure that the armrests are correctly positioned.
6. Do not make any modifications (mechanical alterations) to the wheelchair in order not to compromise its safety in use.
7. Always respect the maximum weight of the wheelchair of 150 Kg.
8. The chair shall be provided with backrest, seat parts, detached footsteps. Insert the two ends of the tubes of back and seat.
9. Assemble the footrests on to tubes of front wheels.
10. Put bucket into bottom of seat.
11. Insert arm into seat tubes.

MAINTENANCE

1. Replace the wheels when they show damage such as cracks or the relief disappears.
2. Periodically check that the bolts are tight and secure.
3. If the brakes are damaged or not working properly, contact your dealer.
4. Routinely clean and disinfect once a day by medical personnel or users, and thoroughly disinfect the chair at patient changeover.
5. Do not put in contact with strong acids and alkalis to avoid corrosion.

WARNINGS

1. Do not use escalators to move a wheelchair while the user is seated. Serious bodily injury may result.
2. Do not reach for objects in any of the following situations:
 - a. If it involves the user having to move forward in the seat.
 - b. If the user must pick them up from the floor
3. Once the footrest has been adjusted, it is important to check that the height and tilt setting does not cause friction between the back of the legs and the upholstery of the chair.
4. The footrests should be at least 65 mm off the floor to prevent possible falls.
5. The brakes should be locked when the user sits down or gets up from the chair.

STORAGE

The product should be stored in normal temperature and humidity and keep dry.

SHELF LIFE

2 years.

WASTE DISPOSAL

Please dispose of the product after use to comply with local regulations.

WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

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