
# EVOLUXION HEALTH

## Green visco

## GeliTech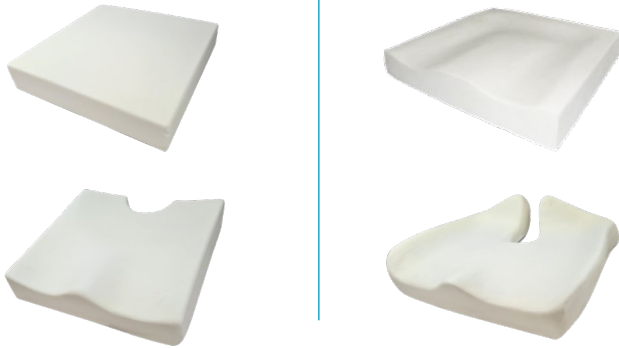

ALMOFADAS ANTI ESCARAS – MANUAL DE UTILIZAÇÃO  
COJINES ANTIESCARAS – MANUAL DE USUARIO  
ANTIDECUBITUS CUSHION– USER MANUAL

COUSSINS ANTI-ESCARRES – MANUEL UTILIZATEUR  
CUSCINI ANTI-ULCERA – MANUALE UTENTE  
ANTI-DEKUBITUS KISEN – BENUTZERHANDBUCH

Last update: 03/10/2024

GeliTech

Evoluxion Health, Lda

## Cojines Antiescaras:

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## Antidecubitus Cushion:

The antidecubitus cushion prevent the appearance of pressure ulcers, consequence from the permanence of the body in the same position for long periods of time. Pressure ulcers appear mainly in areas of bony prominence and support points of the body, such as heels, gluteus, shoulders, knees, elbows and pelvis. Excessive pressure on a point of the body, limits the intake of blood to that area and consequently oxygen, causing redness, destruction of the dermis/epidermis and consequently superficial or deep wounds.

To avoid the appearance of pressure ulcers, caregivers should use different solutions: Regularly change the positioning of the body; Use cushions or devices that minimize pressure at each point of support; Use moisturizing skin creams.

## Model Selection and Usage Indications:

According to the risk of Pressure ulcers (Braden scale or Norton) and weight of the user.

EvoluXion seating cushions are suitable for both wheelchairs and conventional seats, such as sofas, chairs, seats, etc.

## Classification:

EvoluXion seating cushions are considered Class I medical devices, in accordance with Directive 2017/745/EU.

Evolution Health, Lda  
Rua Hermano Neves, 18 – 3º Piso – E7  
1600-477 Lisboa  
Portugal  
[export@evolutionhealth.com](mailto:export@evolutionhealth.com)  
[www.evolutionhealth.com](http://www.evolutionhealth.com)

Model	Dimensions	REF.	Max weight user	Indications and composition	Degree of Risk*
EvoVisco Square	42X42X8 cm	EV805	120 kg	Wound Prevention. Injected Memory Foam 80Kg/m3 density. Cover: PU fabric. Superior part welded.	Medium/High
EvoVisco 42/45	42X42X8 cm 45x42x8 cm	EV800 EV815	120kg	Wound Prevention. Injected Memory Foam 80Kg/m3 density. Cover: PU fabric. Superior part welded.	Medium/High
EvoVisco Coxis Free	42X42X8 cm	EV810	120kg	Wound Prevention. Injected Memory Foam 80Kg/m3 density. Cover: PU fabric. Superior part welded.	Medium/High
Gelitech EvoDuoGel	42X42X8 cm	EV710	120kg	Prevention of bedsores. HR Foam with gel bag of 36x36 cm. Cover: PU fabric. Superior part welded.	High

## Cleaning and Maintenance:

### Cover:

All cushions bring a removable cover, which allows you to wash. It can be washed by hand with cold water, using non-abrasive products or can be washed in the machine, using a program for delicate, cold and centrifuge-free clothing. It is recommended to use a safety net to avoid deformation of the cover during washing.  
Dry in the Horizontal position. Do not use bleach. Don't iron.

\* - Braden or Norton Tables

**Cushion Core:** Viscoelastic foam block - Do not wet.

**Safety:**

Make sure that the cushion fits the seat properly.

**Storage:**

Store the cushion preferably horizontally, avoiding sun exposure and moisture.

**Precautions:**

- Avoid contact of the cushion with solvents, corrosive products and sharp objects.
- Regularly check the normal wear and tear of the cushion, such as appearance modifications, possible snares and/or holes.
- Avoid prolonged contact with liquids.
- Avoid sun exposure;
- Avoid moisture;
- Regularly carry out a visual control of the condition of the cushion, as the aging and wear of the cushion compromises the properties of the cushion, and consequently its function of preventing pressure ulcers.

**Warranty:**

Period of 3 years from the date of purchase. The warranty does not cover the normal wear and tear of the product, as well as inappropriate use of the product. For complaint, contact the point of sale where the cushion was purchased, being mandatory, the presentation of proof of purchase with date and the information of the Article Lot.

**Symbols and Legend:**

	Attention. Read the Instructions first
	Manufacturer
<b>LOTE</b>	Lot
<b>REF</b>	Reference within the EvoluXion Range
	Read the Instructions first.
<b>CE</b>	Certified in accordance with Directive 2017/745/EU

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