POST-SURGERY EYE CUSHION EV625

Purpose:

Sleeping pillow prevents and relieves pressure on the eyes and the appearance of bedsores, providing adequate support during rest. The ergonomic design guarantees a satisfying rest for who spends a lot of time lying down or who is recovering from eye surgery.

Features:

Hollow silicone fiber

Breathable fabric (30% bamboo, 70% polyester)

Function: Positioning and prevention

Size: 50 x 40 x 11 cm

Batch: See package

Color: White

Date of manufacture: See package

Model: EV625

Brand: EcoSoft

Maintenance and storage:

It can be washed by hand with cold water, using non-abrasive products or can be washed in the machine, using a program for delicate, cold and centrifuge-free clothing. It is recommended to use a safety net to avoid deformation of the cover during washing.

Dry in the Horizontal position. Do not use bleach. Do not iron.

Instructions for use:

The antidecubitus cushion prevents the appearance of pressure ulcers, consequence from the permanence of the body in the same position for long periods of time. Pressure ulcers appear mainly in areas of bony prominence and support points of the body, such as heels, gluteus, shoulders, knees, elbows and pelvis. Excessive pressure on a point of the body, limits the intake of blood to that area and consequently oxygen, causing redness, destruction of the dermis/epidermis and consequently superficial or deep wounds.

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