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DESCRIPTION

The raised toilet seat can be used for people with a variety of disabilities or mobility difficulties.

A raised toilet seat is used to provide extra height to enable people to safely get on/off the toilet.

INSTALLATION INSTRUCTIONS

1. To sit on the raised toilet seat, move backwards until you can feel the seat behind your legs. Utilize a grab rail, if available, and lower yourself down.
2. When getting off the raised toilet seat, ensure your feet are firmly placed on the floor, utilize a grab rail, if available, and stand leading with your head and shoulders.

WARNINGS

1. Ensure the Raised Toilet Seat (RTS) is secure and does not move before using.
2. Ensure that you regularly clean the RTS.
3. After cleaning, always re-check the RTS to ensure it is secure.
4. Should the equipment appear damaged DO NOT USE. It may require replacing.

STORAGE

The product should be stored in normal temperature and humidity and keep dry.

SHELF LIFE

5 years.

WASTE DISPOSAL

Please dispose of the product after use to comply with local regulations.

WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

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