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DESCRIPTION

The raised toilet seat can be used for people with a variety of disabilities or mobility difficulties.

A raised toilet seat is used to provide extra height to enable people to safely get on/off the toilet.

ASSEMBLY

Insert the side brackets:

- 1- Turn the toilet lift upside down and place it on a flat surface.
- 2- Insert the brackets into the side guides. Push lightly and turn the screw wheel clockwise to fit the brackets into the toilet riser.

Note: Don't tighten the brackets all the way at this stage, otherwise you won't be able to position the toilet lift on the toilet you want to install it on.

3- Place the toilet lift in the desired toilet. Now tighten the side adjustments you fitted earlier. Use the side adjustment wheel to tighten them until the toilet lift is securely fitted.

Note: Do not overtighten to avoid breaking the parts. The side locking system is designed for easy, tool-free adjustment.

MAINTENANCE AND CLEANING

- Clean regularly by hand with a damp cloth
- Do not use aggressive cleaning agents
- Do not use abrasive cleaning agents
- Do not use cleaning agents with a high chlorine or vinegar content
- Check screws regularly and retighten if necessary.
- The product must not be subjected to temperatures above 60 °C for long periods (e.g. in saunas, etc.).

INSTALLATION INSTRUCTIONS

1. To sit on the raised toilet seat, move backwards until you can feel the seat behind your legs. Utilize a grab rail, if available, and lower yourself down.
2. When getting off the raised toilet seat, ensure your feet are firmly placed on the floor, utilize a grab rail, if available, and stand leading with your head and shoulders.

WARNINGS

1. Ensure the Raised Toilet Seat (RTS) is secure and does not move before using.
2. Ensure that you regularly clean the RTS.
3. After cleaning, always re-check the RTS to ensure it is secure.
4. Should the equipment appear damaged DO NOT USE. It may require replacing.

STORAGE

The product should be stored in normal temperature and humidity and keep dry.

SHELF LIFE

5 years.

WASTE DISPOSAL

Please dispose of the product after use to comply with local regulations.

WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

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