## MANUAL DE USUÁRIO - ES

### Cojines Antiescaras:

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### USER MANUAL - EN

#### Antidecubitus Cushion:

The antidecubitus cushion prevent the appearance of pressure ulcers, consequence from the permanence of the body in the same position for long periods of time. Pressure ulcers appear mainly in areas of bony prominence and support points of the body, such as heels, gluteus, shoulders, knees, elbows and pelvis. Excessive pressure on a point of the body, limits the intake of blood to that area and consequently oxygen, causing redness, destruction of the dermis/epidermis and consequently superficial or deep wounds. To avoid the appearance of pressure ulcers, caregivers should use different solutions: Regularly change the positioning of the body; Use cushions or devices that minimize pressure at each point of support; Use moisturizing skin creams.

Selection of models and indications for use:

According to the risk of ulceration (Braden or Norton) and weight of the user.

The cushions of the EcoSoft EvoluXion line have models suitable for:

1) Seats: Wheelchairs, conventional chairs, sofas, cars, etc.

2) Bed: Positioning and ulcer prevention.

#### Classification:

EvoluXion seating cushions are considered Class I medical devices, in accordance with Directive 2017/745/EU.

Model	Dimensions	REF.	Max weight user	Indications and composition	Degree of Risk*
Square Cushion	44X44X11 cm	EV600	80 kg	Comfort and prevention of bedsores. PU cover, waterproof, breathable and washable. Filled with conjugated siliconized hollow fiber. It has straps to attach the cushion to the wheelchair or other seats.	Medium/Low
Horseshoe Cushion	44X44X11 cm	EV605	80 kg	Comfort and prevention of bedsores. PU cover, waterproof, breathable and washable. Filled with conjugated siliconized hollow fiber. It has straps to attach the cushion to the wheelchair or other seats.	Medium/Low
Round Cushion with Hole	Ø44X11 cm	EV610	80 kg	Comfort and prevention of bedsores. PU cover, waterproof, breathable and washable. Filled with conjugated siliconized hollow fiber. It has straps to attach the cushion to the wheelchair or other seats.	Medium/Low
Square Cushion With Hole	44X44X11 cm	EV615	80 kg	Comfort and prevention of bedsores. PU cover, waterproof, breathable and washable. Filled with conjugated siliconized hollow fiber.	Medium/Low
Ear Pillow	50x42x11 cm	EV620	Not Applicable	Positioning and prevention of bedsores. 30% Bamboo, 70% Polyester cover. Conjugated siliconized hollow fiber filling.	Medium/Low
Eye Pillow	45x40x11 cm	EV625	Not Applicable	Positioning and prevention of bedsores. 30% Bamboo, 70% Polyester cover. Conjugated siliconized hollow fiber filling.	Medium/Low

**CLEANING AND MAINTENANCE:** It can be washed by hand with cold water, using non-abrasive products or can be washed in the machine, using a program for delicate, cold and centrifuge-free clothing. It is recommended to use a safety net to avoid deformation of the cover during washing.

Dry in the Horizontal position. Do not use bleach. Don't iron.

\* Tabela Braden ou Norton

### ⚠ SAFETY:

Make sure that the cushion fits the seat properly.

Storage:

Store the cushion preferably horizontally, avoiding sun exposure and moisture.

### PRECAUTIONS:

- Avoid contact of the cushion with solvents, corrosive products and sharp objects.
- Regularly check the normal wear and tear of the cushion, such as appearance modifications, possible snags and/or holes.
- Avoid prolonged contact with liquids.
- Avoid sun exposure;
- Avoid moisture;

Regularly carry out a visual control of the condition of the cushion, as the aging and wear of the cushion compromises the properties of the cushion, and consequently its function of preventing pressure ulcers.

### WARRANTY:

Period of 3 years from the date of purchase. The warranty does not cover the normal wear and tear of the product, as well as inappropriate use of the product. For complaint, contact the point of sale where the cushion was purchased, being mandatory, the presentation of proof of purchase with date and the information of the Article Lot.

### Symbols and Legend:

	Attention. Read the Instructions first
	Manufacturer
	Lot
	Reference within the EvoluXion Range
	Read the Instructions first.
	Certified in accordance with Directive 2017/745/EU

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