EvoCare

EV520

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(Mattress with adjustable pump)

Operating Principles

- Alternating pressure at a sufficient rhythm avoids prolonged vascular compression which is likely to cause tissue hypoxia.
- Alternating pressure adapts the mattress to the patient’s and their position.

Guidelines For Use

The traditionally accepted method of pressure sore control is to turn the client at regular intervals and to observe the areas of risk regularly.

Alternating pressure mattress is intended to compliment this treatment, observation of the client and turning when necessary, should still be considered as part of the treatment.

General

- For patients who are likely to spend more than 15 hours per day in bed.
- For patients who are classed as medium to high risk of developing pressure ulcers.

Installation/Use

1. Unroll the mattress.
2. Position it on the bed.
3. Secure it to the bed with flaps.
4. Hang the compressor (pump) at the end of the bed (foot end) using the hooks.
5. Connect the mattress feed pipes to the compressor (pump) connector by matching the appropriate outlets.
6. Connect the electrical power supply cable to the compressor (pump).
7. Guide it along the bed to the mattress plug socket and plug in.
8. Press Switch to turn on the compressor (pump).
9. Before positioning the patient onto the mattress, wait until the mattress has been fully inflated.
10. Select the pressure corresponding to the patient’s morphology:

| Level | Recommended weight |
|-------|--------------------|
| 1 | 40 kg |
| 2 | 60 kg |
| 3 | 90 kg |
| 4 | 120 kg |
| 5 | 145 kg |

11. The tubular mattress includes the CPR function to empty the mattress faster in case of emergency.

12. One spare tubular cell and a plug are included to stop a cell if necessary.

Safety measures

- **Do take care** when operating this electrical equipment.
- **Do not** leave connected to electrical supply when not in use.
- **Do not** expose the equipment to excessive humidity.
- **Do not** use corrosive cleaning products such as industrial degreasers or acetic solvents.
- **Do not** attempt to relocate this equipment whilst connected to the electrical supply.

Cleaning/ Disinfection

Compressor (Pump):

- **Do take care** to disconnect from the electrical supply before attempting to clean.
- **Do take care** to use a cloth impregnated with a mild detergent solution.
- **Do take care** to follow manufacturers solution concentrations.
- **Do take care** not to exceed manufacturers recommendations, before wiping off cleaning solution.

Use a cloth impregnated with a detergent product solution or a surface detergent / disinfectant with CE mark following the usage concentrations recommended by the manufacturer follow the endurance times.

Maintenance

Compressor (pump) maintenance operations must be strictly carried out by a technician authorized and trained by the importer.

Mattress Characteristics

Specification:

Max. User Weight: 135-145 Kg

Mattress dimensions:

Started size: 190x85x11,5 cm

Materials: Nylon PVC/PU cell, nylon PVC/PU cover

Compressor (pump) Characteristics.

Specification:

Cycle Time: 6 Mins.

Air Output: 5-7 L/Min.

Pressure Range: 30 ~110 mmHg

Electrical Power Supply: 110V/220V,50-60Hz

Protection Against Class II Electric Shocks.

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