
EvoSofy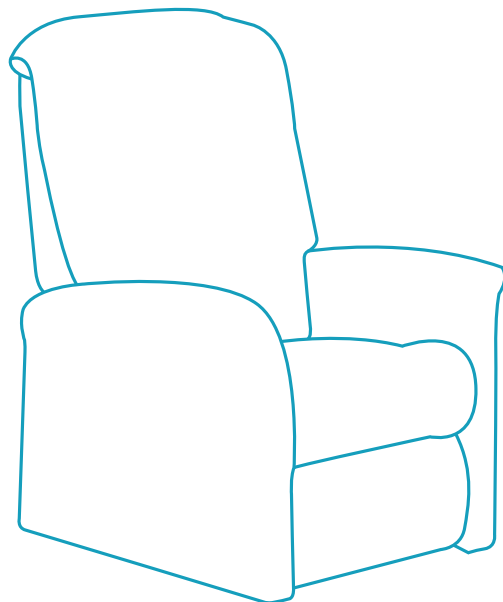

Sillon ortopedico
Orthopaedic sofa
Sofá ortopédico
Sofa orthopédique
Divano ortopedico
Orthopädisches Sofa

Manual de instrucciones
Instruction manual
Manual de instruções
Manuel d'instruction
Manuale di istruzioni
Gebrauchsanweisung

ES - CONTENTS

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EN - CONTENTS

- 1 - Technical Data
- 2 - Installation and mounting
- 3 - Remote control
- 4 - Security
- 5 - Maintenance
- 6 - Cleaning of coating
- 7 - Warranty

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EN - Instructions to the specialist dealer:

This instruction manual is part and parcel of the product and must accompany every product sold.

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EN - First of all we wish to thank you for the trust you placed in us by selecting a **EVOLUXION** product. Before using the chair, read the instruction manual carefully: it will familiarise you with this product. If you have any further questions, please consult your specialist dealer.

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EN - Double motor with foot extension. Vibration massage. Seat heating. Transport wheels with locking and unlocking rear bar for easy movement.

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EN - Double motor, foot extension, vibration massage and lumbar heating.

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EN - Use of buttons:

- 1 motor (leg rest and backrest moves together)

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EN - LIFT FUNCTIONS

Stepless adjustment from sit-position to lift-position.

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EN - LAY BACK FUNCTION

Stepless adjustment to stretched position

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EN - When the liftchair is not being used, leave it the sitting position and not in the elevated position.

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EN - Use of buttons:

- 2 motors (legrest and backrest moves separate)

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1 = Backrest inclination forwards

2 = Backrest inclination backwards

3 = Lift function (stepless adjustment from sit-position to lift-position: seat upwards + legrest downwards)

4 = Lay function (stepless adjustment to stretched position: seat downwards + legrest upwards)

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EN

1. Lift Chair - Pushing the top part of the lift button will close the footrest and lift the chair up.

2. Lower chair - Pushing the bottom part of the lift button will lower the chair and open the footrest.

3. Home - Bring your chair to the home position by pushing this button.

4. Air massage modes - Push this button to change the air massage mode. Push again to cycle through the modes.

The LED signify which mode is selected:

Red - Alternating zones

Green - Pulse

Blue - Wave

White - Massage is off

5. Air massage on/off - Turns air massage zones and modes on and off when pressed. If on, heat will turn off when the air massage is turned off. It will not turn on if the power button is pressed again.

6. Heat on/off

7. Air massage zones - Customize your massage by choosing which zones are activated.

Activated zones will be lit brighter than deactivated zones. If all zones are deselected, the massage will turn off.

8. Furniture adjustments - Using the horizontal rockers, adjust the position of your furniture's headrest, back, lumbar, and footrest.

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EN - Safety

→ Please do not exceed the maximum users weight.

→ Be extremely careful in the presence of children or pets.

Do not allow children to be under the chair, use the chair or play with it because it might cause injuries when the legrest are folding.

→ If the chair blocks in the lying position, you must ask assistance to leave the chair. Please do not place the chair near any heat source or type of electric radiator, fireplace, radiator or radiator with flammable liquid because the coatings are not fire-resistant. Never smoke when seated in the chair.

→ For prolonged use of the liftchair motor (about 2 min), you'll have to wait a few minutes before reusing it. This is needed to avoid overheating the liftchair motor. It is strongly recommended not to operate the chair near a desk or a wall too close to it, as not to cause damage by tilting the back or the legrests.

→ Do not move the liftchair:

- when the liftchair is in the lying position;
- when the liftchair is connected to the power and/or in service.

→ Be careful not to sit on the armrests or on the legrest.

→ Ensure that no cables become squeezed.

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EN - CLEANING THE COATING

You can clean the artificial leather parts with warm water. In case of stubborn dirt, you can wash it with a non-abrasive cleaning product but do not use solvents or hard brushes. You can remove stains with a sponge or soft brush.

We strongly advise you to first test the cleaning product on a hidden part of your liftchair. Always avoid wet surfaces, especially the fabric.

Never wash your liftchair with plenty of water.

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EN - WARRANTY

Defects must be reported in writing within 2 weeks. Manufacturing defects will be under our responsibility starting from the date of the invoice over a period of 3 years. Not included in the warranty:

- Damage due to changes in the structure of our product;
- Wear;
- Damage caused by the user;
- Damage caused by the usage of non-original parts.

The manufacturer takes no liability for damages due to non compliance of these instructions.

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