Dear customer, we thank you for your trust in the products of EMO. By choosing an orthotic of the Corsex line of Hyperextension acquired a qualitative medical product. Read these instructions carefully. If you have any questions, please contact your doctor or your specialist.

REGULATIONS

The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

DESCRIPTION AND CHARACTERISTICS

Paralyzing Ortec MQ401 wrist: Wrist band with Velcro closure. Stabilizes the wrist joint, inhibiting pain during scheduled rest periods. It incorporates a plate made of aluminum and removable. Paralyzing Ortec MQ403 wrist: Adjustable elastic wrist with Velcro closure. Stabilizes the wrist joint, inhibiting pain during scheduled rest periods. It incorporates a plate made of aluminum and removable. Has a wide adjustable strap that improves the immobilization of the joint. Ortec MQ410 Paralyzing bilateral wrist: bilateral elastic wrist with Velcro closure. Stabilization of the wrist joint, inhibiting pain during scheduled rest periods. It incorporates a plate made of aluminum and removable. Has a wide adjustable strap that improves the immobilization of the joint. Ortec MQ405 thumb abduction Wrist: Elastic wrist with Velcro closure. It stabilizes the joint in abduction trapezometacarpal, inhibiting pain during scheduled rest periods. Features a conformable aluminum plate over the joint and removable trapezometacarpal. Ortec MQ415 thumb abduction wrist palmar strap: Elastic wrist with Velcro closure. It stabilizes the joint and wrist joint in abduction trapezometacarpal, inhibiting pain during scheduled rest periods. It incorporates a plate made of aluminum and removable wrist joint and the other on trapezometacarpal. Ortec ART100 Bracing for arthritic hand: Incorporates correction tapes to control ulnar deviation of fingers. Improves trophism and function

of the fingers. Analgesic provides therapeutic heat. Velcro closure and adjustment.

INDICATIONS

Paralyzing Ortec wrist MQ401, MQ403 Ortec adjustable wrist Paralyzing Ortec MQ410 Paralyzing bilateral wrist: Injuries to the wrist-joint without fracture controlled immobilization of the joint, postoperative synovitis, degenerative osteoarthritis, recurrent sprains wrist. Ortec MQ405 wrist abduction thumb trapezometacarpal Osteoarthritis of 1 finger, acute and chronic inflammation of the joint. MQ415 Ortec wrist abduction of thumb strap palmar: Injuries to-articular wrist without a fracture, immobilization controlled joint, post-operative synovitis, osteoarthritis, degenerative, sprains recurrent wrist osteoarthritis trapezometacarpal of 1 finger, acute inflammation and chronic joint. Ortec ART100 Bracing for arthritic hand: Controls the ulnar deviations arthritic fingers, stabilizing the metacarpophalangeal joints and radio-ulnar, preventing the progression of the deformity.

INSTRUCTIONS FOR USE AND INSTALLATIONS

The product should be adjusted by the prosthetist or the physician individually for each patient. The correct choice of size is essential for maximum effectiveness of treatment and prolong the life of the garment. The wrist brace thanks to their design, are easy to install. The patient can be removed and replaced without moving the wrist just the tip, providing rehabilitation exercises and personal hygiene. Adapt the steels to the morphology. Place the brace on the area to the correct height for use. Help of velcro straps to close and adjust the garment.

PRECAUTIONS

Follow the manufacturer's directions for proper orthosis placement. Follow the instructions or recommendations of the prescriber or healthcare personnel. For a personalized adaptation, consult a specialist. If there is ongoing pain, it is advisable to contact a healthcare professional. The used material is hypoallergenic but if there is any

allergic reaction or irritation remove the product and consult with the prescribing doctor. The user should position itself in such a way that the compression is not excessive. In case of doubt about the size, choose the bigger size. Only use during active periods. Remove during the night. The correct choice of the appropriate size for each patient is essential to obtain the highest degree of therapeutic efficacy. The adjustments of the orthotic should be made individually. The orthotic should be used for the purpose it is prescribed for. Do not expose the orthotic to a flame, heat source or high temperatures.

WASHING AND MAINTENANCE

Textile materials: Hand wash with warm water (maximum 30 °) and neutral soap. Dry at room temperature. Do not store or iron the garment. Contact with ointments, creams, alcohols and solvents can deteriorate the garment. Plastic and metal parts: Wash with a cloth moistened with water, do not use solvents or other chemicals. Then dry mainly areas containing metal parts.

QUALITY CERTIFICATES

All the used materials have been tested, comply with the applicable European regulations, are of high quality and offer maximum safety and comfort during use. Öko-Tex Standard 100 Certified: awarded with the Textile eco-label by completing a catalogue of criteria that ensure the absence of harmful substances. Certifies the absence of harmful substances in all materials with which the orthotic is made.

WARRANTY

Especialidades Médico Ortopédicas S.L., guarantees that all its products, as long as they have not been manipulated or altered, always are within the framework of the legal terms. Products whose characteristics have been altered due to misuse

are not included in this warranty.

NOTICE

To dispose of the product, use the solid waste container. For the disposal of the container, depending on its material, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of the country. Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

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