
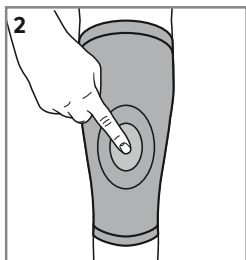
JuzoFlex

Genu Xtra

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użyci · Инструкция по эксплуатации · تعليمات الاستخدام · Návod na používanie · Navodila za uporabo

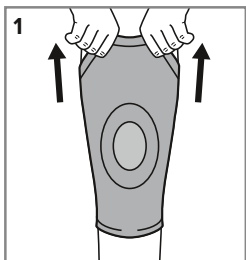Ziehbekimmung

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Purpose

JuzoFlex knee supports provide anatomical guidance for the affected knee joint, thus relieving it during movement. Medical compressive active supports have a proprioceptive effect. They improve self-perception and can compensate for muscle imbalances. They reduce pain, support the removal of fluids from the tissue, thanks to their rheological effect, and provide relief during movement.

Here is how to put on your Juzo Knee Support correctly:

1. Pull the knee support over your forefoot up to the right place on your knee.
2. Then position the knee support so that the patella (kneecap) is enclosed by the patella pad.

Put on your support while sitting down to prevent injuries or falling. The support fits properly when it surrounds the knee without applying too much (painful) pressure. You can re-adjust the support by repeating step 2 at any time. Caution: rough, unfiled finger nails or sharp-edged rings etc. may damage the knitted fabric. Toe nails and calluses can cause damage.

Please note:

Only wear your support when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the support can only be guaranteed if it is applied correctly. Although your support is largely resistant to oils, ointments, skin moisture and other environmental influences, the fabric as well as worked in pressure pads can be adversely affected by certain combinations of these factors. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the support yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the support is not suitable for treating more than one patient.

Washing and care instructions

Please note the care instructions on the textile label sewn into your compression garment. Wash the supports separately the first time (colour may run). Your support should be washed every day on a gentle or delicate wash cycle (30 °C). Tip: A mesh laundry bag provides additional protection for the knitted fabric. Please do not use fabric softener! When hand washing, rinse well and do not wring. We recommend the gentle Juzo Special Detergent. You can reduce drying time by placing the supports on a thick towel after washing, tightly rolling the towel up and firmly pressing out excess moisture. Then hang the support up loosely. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Juzo supports are not suitable for dry cleaning. The support is not suitable for tumble drying.

Material composition

Details are provided on the label sewn into your knee support.

Storage information and shelf life

Store in a dry place and protect from sunlight. Supports have a maximum useful life of 42 months. The useful life of the medical product is printed on the box with an hourglass symbol. The medical retailer or doctor prescribing the support will advise on exactly how long it can be used for. Body measurements may change based on the patient's condition and individual circumstances. We therefore recommend that medical specialists check body measurements regularly.

Indications

Tendency towards patellar dislocation, post-traumatic oedema, knee joint sprains and effusions, chronic, post-traumatic or postoperative irritation of the soft tissue in the area of the knee, meniscopathy, stretched or weakened ligaments

Contraindications

If you have any of the following conditions, you should only wear the support after consulting with your doctor: Lymph drainage disorders and non-specific soft tissue swelling outside the area of application, sensory disturbances and impaired circulation in parts of the body to be treated, skin conditions, skin conditions, skin irritation and injuries in the area to be treated; open wounds in the treatment area must be covered with sterile material. Liability for failure to comply with the contraindications cannot be accepted by the Julius Zorn GmbH Group.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your symptoms worsen during the time you are wearing the product, please take off the support and consult a medical specialist immediately. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

As a rule, the product is to be destroyed via the conventional disposal route after the end of use.

Please observe the applicable local/national regulations.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany · juzo.com