Manual de Instrucciones

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inclusive solutions
WALKING AIDS

Instructions Manual

CANES, ELBOW CRUTCHES, CRUTCHES, PYRAMIDS

The elbow crutches, canes, crutches and pyramids are external use walking aids which replace partially or in full a human lower limb and are to be used by people with temporary or definitive disabilities, patients, elders and handicaps in their going about on foot. These devices are made up of aluminum tubes with thermoplastic handle, adjustable in height by metal pins having tips of rubber at the ends in contact with the ground. The cane has a handle on PP, the elbow crutch has a hinged or fixed thermoplastic rim. The crutch has a wooden handle and a polyurethane axillar support and the pyramid has a handle on PVC. All these devices are Class I medical devices according to the communitarian directive 2007/47/EC of September 5th.

Thank you for choosing an ORTHOS XXI product.

TO ASSURE A LONGEVITY AND EFFICIENCY OF YOUR WALKING AIDS YOU MUST:

- Be aware that as walking aids, the elbow crutches, canes, crutches and pyramids must not be inappropriately used (hitting doors, throwing objects, impacts or strong hits);
- Replace the tips every time its shows wearing off or deterioration.

SETTING THE DEVICE TO THE USER

The elbow crutch must be adapted so that the arm and the hand are well supported either on the handle or on the elbow crutch rim. The axillary crutch should be adapted so that your hand is well supported in the wrist, and the armpit supported the crutch. Canes and pyramids should be adjusted so that the arm is well stretched in order to ensure good support for the forearm and shoulder, and by these, the entire bone structure, safeguarding degenerative conditions by incorrect posture. The position of elbow crutches, canes, crutches, and pyramids should be substantially vertical and their movement must be made in small steps following the movement of the body to maintain the body always well supported.

SPECIAL CARES

- Do not use elbow crutches, crutches or pyramids away from the body, or poorly supported, to avoid accidents due to imbalance and falls.

Be aware that wet flat surfaces or slippery grounds may cause the user's unbalance and/or fall and with the contact with chemical products because it can prematurely cause the breakdown of the tips.

The irregular grounds (stones, grids on the sidewalks, small steps) may strain the tips or obstruct the walking which can cause unbalance and/or falls. Therefore the elbow crutches, crutches and pyramids should be well raised and supported when going through stony or bumpy grounds.

The soft surfaces are highly dangerous because the tip may bury and cause unbalance to the user.

MAINTENANCE OF THE DEVICE

The device must be clean when necessary with a low abrasive cloth soaked in a neutral detergent diluted in water. Dry the device with a soft cloth. The device can be disinfected with alcohol and derivate, since they are diluted in water. The thermoplastic components should be inspected regularly and replaced when they show wear or fracture. The straps of the inner tube should be regularly lubricated with

Vaseline.

SETTING THE HEIGHT OF DEVICE

Whenever you need to adjust the height of the device, proceed as follows:

- 1st Remove, adjust and introduce the upper bracket (3) in the bore suitable for height adjustment of device;
- 2nd Entering the inner tube with the spring placed as in figure, in cases including damping;
- 3rd Place the inner bracket (3') below two holes from the first one:
 - If this traverse the oval hole (6) it has the damping effect;
 - If this pass through the round hole (7) it cancels the effect of damping.

Where necessary adjust the height of elbow crutch without damping, proceed as follows:

- 1st Remove the clamp (3);
- 2nd Adjust the elbow crutch to the desired height;
- 3rd Replace the clamp (3).

SUBSTITUTION OF THE TIPS

Whenever the tip presents waste, it should proceed to its substitution, following the indications:

- 1st Remove the used tip;
- 2nd Place the new tip.

PRESCRIPTION SKILLS

The prescription skills to evaluate if the patient, handicap or elder are fitted to use these walking aids are only assured by the medical staff such as doctors, physiatrists, orthopaedist, nurses or other health professionals.

In alternative or when these are not available the pharmacy auxiliary staff, the official orthopaedic materials' salesmen, the assistants of home for elders or the patient's relatives, after a careful read through of the instructions guide, may advise about the adjust in height, check the deterioration of the tips and replace these when they are wear off.

These walking aids may help those who need them to be less dependent, freer, to do their daily walks, go to work, leave their houses, have fun and live in society, becoming therefore more useful despite their small problems.

MANUFACTURER WARRANTY

This product has two years warranty after the purchase's invoice date, covering production imperfections. The warranty does not cover the wearing out produced by a normal utilization of the device, as well as defects and damages related with improper manipulation or use or inadequate maintenance.