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Instructions Manual WALKING STICKS

ORTHOS XXI products are equipped with studied equipments in order to serve de patients, handicaps and elders, with multiple handicaps; its utilization must be prescribed by health professionals or authorized salesmen who are familiar with the products, what they are for and the instructions manual.

Our wide experience and manufacture technology allow us to adjust the products according to the physical handicaps of the final user. The walking sticks are a medical devide according to the communitary directive 2007/47/EC of September 5th and tested according to the ISO 11199-1:1999.

Thank you for choosing an ORTHOS XXI product.

MAIN FEATURES

Orthos XXI's walking sticks are external use walking aids which replace partly or fully a lower limb and which are to be used by people temporarily or permanently handicapped, patients or elders in their go about on foot.

The walking sticks are made of an aluminium tube with a handgrip adjustable in height by metallic pins, with rubber tips in the end touching the walking ground.

The walking sticks are prepared for a maximum load of 125 kg and height to 1,80 m. The child model is suitable for a maximum weight of 100 kg and a maximum height of 1.55 m.

TO ASSURE THE LONGEVITY AND EFFICIENCY OF YOUR WALKING STICKS YOU MUST:

- Be aware that the walking stick must not be inappropriately used (hitting doors, throwing objects, impacts or strong hits).
- Replace the tips every time its appearance shows wearing off or deterioration.

WAY OF USE

- Adjust the height of your walking stick correctly according to your size.
- Keep in mind that when the walking stick is used away from the body or badly supported may cause accidents by unbalance and falls.
- The walking stick must be held by the handle grips placed on the sides.
- ATTENTION: The walking stick must have all four tips correctly resting on the walking ground at the same time in order to avoid unbalance.** To walk with the fix model raise the walking stick and move it slightly forward; with the articulated model walk strolling the walking stick side parts with the body. In all models the walking must occur in small steps and according to body movements.

- The models ANFFWZZ01Q and ANARWZZ01Q allow to fold the side parts of the walking stick in order to easy its' transport.

SPECIAL CARES

- Be aware that wet flat surfaces or slippery grounds which may cause the instability of the walking and the user's unbalance and/or fall.
- Pay special attention when walking around through grounds in contact with chemical products because it can corrode prematurely the tips.
- The irregular grounds (stones, grids on the sidewalks, small steps) may strain the tips or obstruct the walking which can cause unbalance and/or falls. Therefore walking stick should be well raised and supported when going through stony and bumpy grounds.
- The soft surfaces are highly dangerous because the tip may plunge and cause unbalance to the user.

MAINTENANCE OF THE DEVICE

The device must be kept clean in order to increase its lifetime. The device must be washed when necessary with a low abrasive cloth or sponge soaked in water or a neutral detergent (pH between 5 and 8) diluted (5%) in water. Dry the device with a soft cloth. The thermoplastic components should be inspected regularly and replaced when they show wear or fracture.

PRESCRIPTION SKILLS

The prescription skills to evaluate if the patient, handicap or elder are fitted to use these walking aids are only assured by the medical staff such as doctors, physiatrists, orthopaedist, nurses or other health professionals. In alternative or when these are not available the pharmacy auxiliary staff, the official orthopaedic materials' salesmen, the assistants of home for elders or the patient's relatives, after a careful read through of the instructions guide, may advise about the adjust in height, check the deterioration of the tips and replace these when they are wear off.

The use of walking sticks occurs by necessity, not by the equipment itself but by the inabilities of its user. These walking aids may help those who need them to be less dependent, freer, to do their daily walks, go to work, leave their houses, have fun, live in society, and practice physical activities becoming therefore more useful despite their small problems.

Model	Maximum dimensions of the base (cm)	Height minimum/maximum (cm)
52 AN W FX 01	56x43	86 / 98
52 AN W FX 02	47x39	79 / 89
52 AN X AR 01 2N	59x48	82 / 92
52 AN W AR 01	59x48	82 / 92

EN ISO 11199-1:1999

MANUFACTURER WARRANTY

This product has two years warranty against production imperfections since purchase. The warranty does not cover the wearing out of the tips or other wear and tear material and will not cover problems related with improper use or inadequate maintenance.

The transport costs for the repair are of the clients' responsibility.