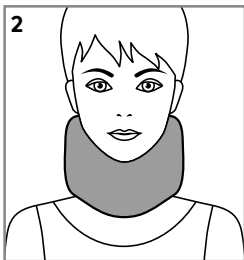
JuzoPro

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Intended purpose

The JuzoPro Cervical Collar Supports stabilize the

cervical spine thanks to their contours and potential reinforcing elements.

This is how to put on your JuzoPro Cervical Collar Support correctly:

Open the orthosis fully and place it around your neck so that the fastener (velcro strap) is located at the back of your neck.

When it is positioned correctly, your chin should be embedded in the opening (recess) provided for it. The orthosis is especially designed for easy adjustability so you can accommodate the right tightness to help stabilise the neck, and prevents restriction. You can re-adjust it at any time if necessary.

Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realised if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient.

Washing and care instructions

Please observe the care instructions on the textile label attached to your orthosis. Wash the orthosis separately the first time (the colour might come out). Close the velcro fastener and wash the orthosis by hand at approx. 30 °C using a laundry detergent for delicate fabrics. We recommend using the mild Juzo Special Detergent for this. Afterwards, please rinse out the orthosis well and allow it to dry at room temperature. To reduce the drying time, you can place the orthosis on a thick terry towel and pat it dry. Do not leave in the towel and do not place on a heater or in the sun to dry. This Juzo product must not be cleaned chemically.

Material composition

The exact details can be found on the textile label on your orthosis.

Storage information

Store in a dry place and protect from sunlight.

Indications

Cervical syndrome, whiplash, torticollis, post-traumatic pain, osteochondrosis (degenerative change), spondylarthrosis (wear of the cervical spine)

Contraindications

If the following symptoms exist, the orthosis should

only be worn after consultation with the physician: tracheostomy, hypersensitivity in the neck area, severe swelling, skin disorders or skin irritations; Open wounds in the treated area should be kept sterile. The Julius Zorn Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your symptoms worsen during the time you are wearing the product, please take off the orthosis and consult a medical specialist immediately. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

In the event of complaints related to this product, such as damage to the material or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

Your orthosis can be disposed of through normal waste channels. No special waste disposal requirements apply.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany · juzo.com