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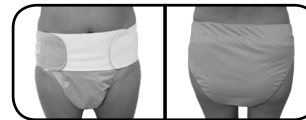
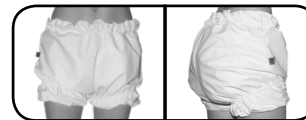
EN

DESCRIPTION

Pyjamas, nightdresses, incontinence knickers and restraint sheets are included within the incontinence category of our company.

These products are so comfortable because they prevent users to access to the diaper. Therefore, the products improve the daily tasks of health professionals (there's no need to change the user's clothes or bedding).

This product is indispensable for people suffering from any kind of cognitive deterioration because costs are considerably reduced: less cleaning, less time of cleaning of patients or facilities, etc. Furthermore, the appearance of users will be considerable improved.

INDICATIONS

Our company, Ubiotex Quality life, has developed a huge range of incontinence products.

CLASSIFICATION

UBIOTEX INCONTINENCE PRODUCTS are considered a Type I medical device (rule 1). Conformity evaluation was carried out by following the indications in Annex VII (Declaration CE of Conformity) of the Directive 93/42/CEE and the Directive 2007/47.

In accordance with Annex IX of the Directive 93/42/CEE and its subsequent modifications, the health product is defined as:

- Non-Invasive Health Product

Rule 1.- All non-invasive products are included in the Type I category unless one of the following rules is applicable.

MODELS

- 1.- INCONTINENCE PYJAMAS
- 2.- INCONTINENCE NIGHTDRESSES
- 3.- INCONTINENCE KNICKERS
- 4.- RESTRAINT SHEETS

CLEANING/DESINFECTATION

1. INCONTINENCE PYJAMAS AND NIGHTDRESSES:      

Hand and machine washable at a temperature of 60°C. Dry clean product. This product can be ironed. Do not use bleach.

2. INCONTINENCE KNICKERS:

•Polyurethane fabric:    

Hand and machine washable at a temperature of 110°C. Dry clean product. Do not use abrasive products.

•Sanitized fabric:      

Hand and machine washable at a temperature of 60°C Dry clean product. Do not iron. Use dryer. Do not use bleach.

•Knicke-Diaper waterproof:       

Hand and machine washable at a temperature of 60°C Dry clean product. Do not iron. Use dryer. Do not use bleach.

3. RESTRAINT SHEETS:      

Hand and machine washable at a temperature of 60°C. Dry clean. This product can be ironed. Use dryer. Do not use bleach.

INCONTINENCE PRODUCTS

EN

SECURITY

Verify that the size of the product fit the size of the user. Loose-fitting product (especially incontinence knickers with elastic straps).
The manipulation of the product must be supervised by a health professional.

PRECAUTIONS

Do not use caustic, abrasive, solvent or sharp objects in direct contact with the product.
Test regularly the progressive cushion's deterioration (orifices, cracks, surface's modification, etc.)
Take precautions for every material used in the products.

MATERIAL'S PRECAUTIONS

Avoid prolonged exposure to liquids and humidity conditions with the product.

MAINTENANCE

Store the cushions preferably in a flat position and protect against sunlight and humidity.
Keep the product inside its original package until the moment of its use.

RECYCLING

Do not burn nor throw away in the nature, outside the correct places for this action.

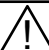
GUARANTEE

The guarantee period of all cushions is two years from the date of the original purchase.
This guarantee does not cover the regular deterioration of the product and does not substitute legal guarantees.
Please contact the Quality Department of the company to any incident that may occur to correctly resolve it.

LIFETIME OF THE PRODUCT

The regular lifetime of the products is about a year.

LEGEND AND TAG'S SYMBOLS

	Manufacturer.
	Serial number.
	Catalogue number.
	Read the directions for use first.
	Certificate in accordance with the Directive 93/42/EEC.
	Attention. Read the directions for use first.

MODEL		SIZE / MEASURE	REF.	INSTALATION	INDICATIONS FOR USE	COMPOSITION
PYJAMAS	ALL SHORT	S / Up to 85 cm	201100	<p>The product disposes of two zippers.</p> <p>One is placed between the legs and the other on the back.</p> <p>Open the zippers and get the head through the pyjama.</p> <p>Close the zipper of the back and the one of the legs.</p>	<p>Maximum comfort of the user.</p> <p>The product improves the daily tasks of health professionals (change the diaper or clean the patient) thanks to the zipper placed between the legs.</p> <p>Day and night use.</p> <p>The short model can be used as underwear. The long model can be used as a pyjama.</p>	<p>Sanitized: Pique (50% cotton, 50% polyester) Zipper Velcro</p>
		M / Up to 100 cm	201102			
		L / Up to 108 cm	201104			
		XL / Up to 120 cm	201106			
		XXL / Custom-fitted	201108			
	SHORT SLEEVE LONG PANTS	S / Up to 85 cm	201270			
		M / Up to 100 cm	201272			
		L / Up to 108 cm	201274			
		XL / Up to 120 cm	201276			
		XXL / Custom-fitted	201278			
	ALL LONG	S / Up to 85 cm	201200			
		M / Up to 100 cm	201202			
		L / Up to 108 cm	201204			
		XL / Up to 120 cm	201206			
	LONG SLEEVE SHORT PANTS	XXL / Custom-fitted	201208			
		S / Up to 85 cm	201360			
		M / Up to 100 cm	201362			
		L / Up to 108 cm	201364			
		XL / Up to 120 cm	201366			
	ALL LONG WITH FEET	XXL / Custom-fitted	201368			
S / Up to 85 cm		201310				
M / Up to 100 cm		201312				
L / Up to 108 cm		201314				
XL / Up to 120 cm		201316				
ALL LONG WINTER	XXL / Custom-fitted	201318				
	S / Up to 85 cm	201290				
	M / Up to 100 cm	201292				
	L / Up to 108 cm	201294				
ALL LONG WITH FEET WINTER	XL / Up to 120 cm	201296				
	XXL / Custom-fitted	201298				
	REF.	201301				
						<p>Winter: Plush (50% cotton, 50% polyester) Zipper Velcro</p>

	MODEL	SIZE / MEASURE	REF.	INSTALATION	INDICATIONS FOR USE	COMPOSITION					
KNICKERS	SANITIZED	S / Up to 85 cm	204100	<p>Get the legs carefully through the legs.</p> <p>Verify that the product doesn't tighten (emergence of redness due to the strap's pressure).</p> <p>Open the Velcro closures of the sides. Place first the knickers through the back side and then through the frontal side.</p> <p>Adjust the product with the Velcro closure. Verify that the product doesn't tighten (emergence of redness due to the strap's pressure).</p> <p>Get the legs through the short knickers. Verify that the product doesn't tighten (emergence of redness due to the strap's pressure).</p>	<p>Additionally, this product can be used to fasten the diaper and avoid the loss of liquids by its lateral sides. This product is used to protect the user from liquids.</p> <p>In this way, this product is used to fasten the diaper and avoid involuntary movements or unbutton.</p> <p>The use of this product offers the users a huge freedom of movement when the diaper has to be adjusted. Protection against possible losses of liquids.</p>	Sanitized: Pique (50% cotton, 50% polyester) Elastic strap					
		M / Up to 100 cm	204102								
		L / Up to 108 cm	204104								
		XL / Up to 120 cm	204106								
		XXL / + 120 cm	204108								
	WATERPROOF LIGHT BLUE	S / Up to 85 cm	204200			<p>Get the legs carefully through the legs.</p> <p>Verify that the product doesn't tighten (emergence of redness due to the strap's pressure).</p>	<p>Additionally, this product can be used to fasten the diaper and avoid the loss of liquids by its lateral sides. This product is used to protect the user from liquids.</p> <p>In this way, this product is used to fasten the diaper and avoid involuntary movements or unbutton.</p> <p>The use of this product offers the users a huge freedom of movement when the diaper has to be adjusted. Protection against possible losses of liquids.</p>	Polyurethane 100% Elastic strap			
		M / Up to 100 cm	204202								
		L / Up to 108 cm	204204								
		XL / Up to 120 cm	204206								
		XXL / + 120 cm	204208								
	WATERPROOF WHITE	S / Up to 85 cm	204210			<p>Get the legs carefully through the legs.</p> <p>Verify that the product doesn't tighten (emergence of redness due to the strap's pressure).</p>		<p>Additionally, this product can be used to fasten the diaper and avoid the loss of liquids by its lateral sides. This product is used to protect the user from liquids.</p> <p>In this way, this product is used to fasten the diaper and avoid involuntary movements or unbutton.</p> <p>The use of this product offers the users a huge freedom of movement when the diaper has to be adjusted. Protection against possible losses of liquids.</p>	Polyurethane 100% Elastic strap		
		M / Up to 100 cm	204212								
		L / Up to 108 cm	204216								
		XL / Up to 120 cm	204218								
		XXL / + 120 cm	204220								
	WATERPROOF WITH VELCRO LIGHT BLUE	S / Up to 85 cm	204300			<p>Open the Velcro closures of the sides. Place first the knickers through the back side and then through the frontal side.</p>			<p>Additionally, this product can be used to fasten the diaper and avoid the loss of liquids by its lateral sides. This product is used to protect the user from liquids.</p> <p>In this way, this product is used to fasten the diaper and avoid involuntary movements or unbutton.</p> <p>The use of this product offers the users a huge freedom of movement when the diaper has to be adjusted. Protection against possible losses of liquids.</p>	Polyurethane 100% Elastic strap Velcro	
		M / Up to 100 cm	204302								
		L / Up to 108 cm	204304								
		XL / Up to 120 cm	204306								
		XXL / + 120 cm	204308								
	WATERPROOF WITH VELCRO WHITE	S / Up to 85 cm	204312			<p>Adjust the product with the Velcro closure. Verify that the product doesn't tighten (emergence of redness due to the strap's pressure).</p>				<p>Additionally, this product can be used to fasten the diaper and avoid the loss of liquids by its lateral sides. This product is used to protect the user from liquids.</p> <p>In this way, this product is used to fasten the diaper and avoid involuntary movements or unbutton.</p> <p>The use of this product offers the users a huge freedom of movement when the diaper has to be adjusted. Protection against possible losses of liquids.</p>	Polyurethane 100% Elastic strap Velcro
		M / Up to 100 cm	204314								
		L / Up to 108 cm	204316								
		XL / Up to 120 cm	204318								
XXL / + 120 cm		204320									
ADJUSTABLE	Standard	204175		<p>Additionally, this product can be used to fasten the diaper and avoid the loss of liquids by its lateral sides. This product is used to protect the user from liquids.</p> <p>In this way, this product is used to fasten the diaper and avoid involuntary movements or unbutton.</p> <p>The use of this product offers the users a huge freedom of movement when the diaper has to be adjusted. Protection against possible losses of liquids.</p>	Polyurethane 100%. Sanitized: Pique (50% cotton, 50% polyester). Elastic strap & Velcro						
POLOLO WHITE	S / Up to 85 cm	204500	<p>Get the legs through the short knickers. Verify that the product doesn't tighten (emergence of redness due to the strap's pressure).</p>		<p>Additionally, this product can be used to fasten the diaper and avoid the loss of liquids by its lateral sides. This product is used to protect the user from liquids.</p> <p>In this way, this product is used to fasten the diaper and avoid involuntary movements or unbutton.</p> <p>The use of this product offers the users a huge freedom of movement when the diaper has to be adjusted. Protection against possible losses of liquids.</p>	Polyurethane 100% Elastic strap & Velcro.					
	M / Up to 100 cm	204502									
	L / Up to 108 cm	204504									
	XL / Up to 120 cm	204506									

MODEL		SIZE / MEASURE	REF.	INSTALATION	INDICATIONS FOR USE	COMPOSITION
KNICKERS	DIAPER LIGHT BLUE	S / Up to 85 cm	204410	Get the legs through the knickers. Verify that the product doesn't tighten (emergence of redness due to the strap's pressure).	This product can be used in pools. Do not use additional diapers.	Polyurethane Napa Cellulose Vinyle Elastic strap
		M / Up to 100 cm	204412			
		L / Up to 108 cm	204414			
		XL / Up to 120 cm	204416			
		XXL / + 120 cm	204418			
	DIAPER WHITE	S / Up to 85 cm	204400			
		M / Up to 100 cm	204402			
		L / Up to 108 cm	204404			
		XL / Up to 120 cm	204406			
		XXL / + 120 cm	204408			

MODEL		SIZE / MEASURE	REF.	INSTALATION	INDICATIONS FOR USE	COMPOSITION
NIGHTDRESS	SANITIZED	90	202100	The product disposes of a L-shaped zipper. Open the zipper and get the head through the nightdress. Close the zipper placed on the feet zone.	Maximum comfort of the user. The product improves the daily tasks of health professionals (change the diaper or clean the patient) thanks to the L-shaped zipper. Day and night use. The short model can be used as underwear. The long model can be used as a pyjama.	Sanitized: Pique (50% cotton, 50% polyester) Zipper & Velcro
		105	202200			
	WINTER	90	202101			
		105	202201			

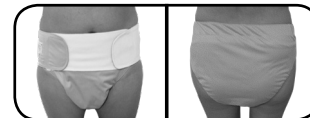
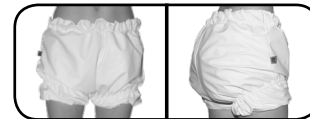
MODEL		SIZE / MEASURE	REF.	INSTALATION	INDICATIONS FOR USE	COMPOSITION				
RESTRAINT SHEETS	SANITIZED SLEEVELESS	90x20x190	203100	<p>Open the zipper and place the restraint sheet on the mattress as a cover.</p> <p>The part of the zipper corresponding to the part of the user must remain on the superficial part of the bed.</p> <p>Place the user on the bed. Get the arms through the hole of the sleeves.</p> <p>Close the zipper from the zone of the chest of the user and close it to the feet of the bed.</p> <p>The sheet disposes of a hole to place the tubes of the catheter or drainages.</p>	<p>Maximum comfort of the user.</p> <p>The product improves the daily tasks of health professionals (change the diaper or clean the patient) thanks to the zipper placed between the legs.</p> <p>Day and night use.</p> <p>The short model can be used as underwear. The long model can be used as a pyjama.</p> <p>The user can't access to the diaper.</p> <p>This product is recommended to those cases where there's a cognitive deterioration and also a danger of escaping/falls. It consists of an alternative form of bed restraints.</p>	Sanitized: Pique (50% cotton, 50% polyester) Zipper Velcro				
		105x20x190	203102							
		135x20x190	203104							
		150x20x190	203106							
	WINTER SLEEVELESS	90x20x190	203350			<p>Open the zipper and place the restraint sheet on the mattress as a cover.</p> <p>The part of the zipper corresponding to the part of the user must remain on the superficial part of the bed.</p> <p>Place the user on the bed. Get the arms through the hole of the sleeves.</p> <p>Close the zipper from the zone of the chest of the user and close it to the feet of the bed.</p> <p>The sheet disposes of a hole to place the tubes of the catheter or drainages.</p>	<p>Maximum comfort of the user.</p> <p>The product improves the daily tasks of health professionals (change the diaper or clean the patient) thanks to the zipper placed between the legs.</p> <p>Day and night use.</p> <p>The short model can be used as underwear. The long model can be used as a pyjama.</p> <p>The user can't access to the diaper.</p> <p>This product is recommended to those cases where there's a cognitive deterioration and also a danger of escaping/falls. It consists of an alternative form of bed restraints.</p>	Winter: Plush (50% cotton, 50% polyester) Zipper Velcro		
		105x20x190	203352							
		135x20x190	203354							
		150x20x190	203356							
	SANITIZED LONG SLEEVE	90x20x190	203200					<p>Open the zipper and place the restraint sheet on the mattress as a cover.</p> <p>The part of the zipper corresponding to the part of the user must remain on the superficial part of the bed.</p> <p>Place the user on the bed. Get the arms through the hole of the sleeves.</p> <p>Close the zipper from the zone of the chest of the user and close it to the feet of the bed.</p> <p>The sheet disposes of a hole to place the tubes of the catheter or drainages.</p>	<p>Maximum comfort of the user.</p> <p>The product improves the daily tasks of health professionals (change the diaper or clean the patient) thanks to the zipper placed between the legs.</p> <p>Day and night use.</p> <p>The short model can be used as underwear. The long model can be used as a pyjama.</p> <p>The user can't access to the diaper.</p> <p>This product is recommended to those cases where there's a cognitive deterioration and also a danger of escaping/falls. It consists of an alternative form of bed restraints.</p>	Sanitized: Pique (50% algodón, 50% poliéster) Zipper Velcro
		105x20x190	203202							
		135x20x190	203204							
	WINTER LONG SLEEVE	150x20x190	203206							<p>Open the zipper and place the restraint sheet on the mattress as a cover.</p> <p>The part of the zipper corresponding to the part of the user must remain on the superficial part of the bed.</p> <p>Place the user on the bed. Get the arms through the hole of the sleeves.</p> <p>Close the zipper from the zone of the chest of the user and close it to the feet of the bed.</p> <p>The sheet disposes of a hole to place the tubes of the catheter or drainages.</p>
		90x20x190	203320							
		105x20x190	203322							
		135x20x190	203324							
	SANITIZED SHORT SLEEVE	150x20x190	203326	<p>Open the zipper and place the restraint sheet on the mattress as a cover.</p> <p>The part of the zipper corresponding to the part of the user must remain on the superficial part of the bed.</p> <p>Place the user on the bed. Get the arms through the hole of the sleeves.</p> <p>Close the zipper from the zone of the chest of the user and close it to the feet of the bed.</p> <p>The sheet disposes of a hole to place the tubes of the catheter or drainages.</p>	<p>Maximum comfort of the user.</p> <p>The product improves the daily tasks of health professionals (change the diaper or clean the patient) thanks to the zipper placed between the legs.</p> <p>Day and night use.</p> <p>The short model can be used as underwear. The long model can be used as a pyjama.</p> <p>The user can't access to the diaper.</p> <p>This product is recommended to those cases where there's a cognitive deterioration and also a danger of escaping/falls. It consists of an alternative form of bed restraints.</p>					
		90x20x190	203250							
		105x20x190	203252							
		135x20x190	203254							
	WINTER SHORT SLEEVE	150x20x190	203256			<p>Open the zipper and place the restraint sheet on the mattress as a cover.</p> <p>The part of the zipper corresponding to the part of the user must remain on the superficial part of the bed.</p> <p>Place the user on the bed. Get the arms through the hole of the sleeves.</p> <p>Close the zipper from the zone of the chest of the user and close it to the feet of the bed.</p> <p>The sheet disposes of a hole to place the tubes of the catheter or drainages.</p>	<p>Maximum comfort of the user.</p> <p>The product improves the daily tasks of health professionals (change the diaper or clean the patient) thanks to the zipper placed between the legs.</p> <p>Day and night use.</p> <p>The short model can be used as underwear. The long model can be used as a pyjama.</p> <p>The user can't access to the diaper.</p> <p>This product is recommended to those cases where there's a cognitive deterioration and also a danger of escaping/falls. It consists of an alternative form of bed restraints.</p>			
90x20x190		203300								
105x20x190		203302								
135x20x190		203304								

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