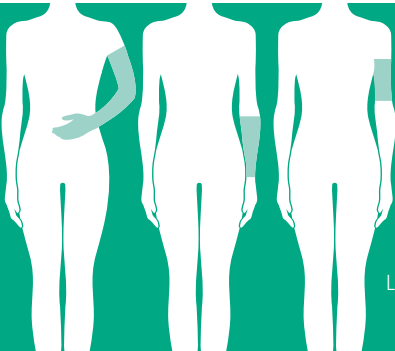
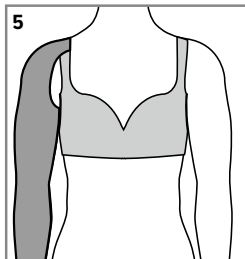
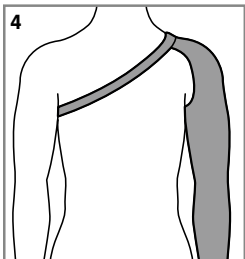
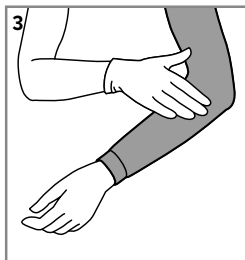
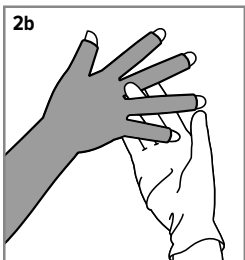
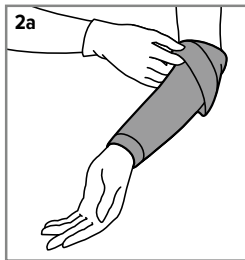
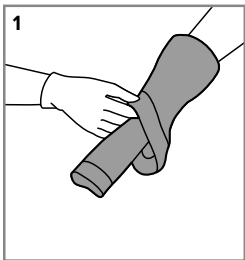
Kompressionsärmel / Armsegmente

Compression sleeves / arm wraps

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użyci · Инструкция по эксплуатации · تعليمات الاستخدام · Návod na používanie · Navodila za uporabo

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EN

Intended purpose

Compression therapy involves measures to apply extensive, external pressure to treat vein problems and lymph drainage disorders. The aim of the extensive pressure is to prevent the formation of oedema and to support venous return and lymph drainage. Compression garments exert an optimal pressure gradient that gradually decreases from bottom to top. Compression therapy is primarily used to treat vein problems and oedema. It involves the use of compressive and elastic double-stretch fabrics or knitted fabrics that wrap right round the extremities. As a general rule, compression therapy is a long-term treatment requiring appropriate products that are used by the patients themselves when medically indicated and on the instructions of a doctor.

Notes for use

Your specialist retailer will gladly show you the correct way to apply the product. You can also visit juzo.com/donning for video instructions relating to all Juzo donning aids. Put on your compression garments in the morning straight after you get out of bed. Rough, unfiled finger nails or sharp-edged rings etc. may damage the very fine knitted fabric. We therefore recom-

mend the use of **Juzo special gloves**. In order to increase the wearing comfort of your compression garments, you can hoist the silicone border a little (if there is one) a number of times during wearing.

Compression sleeves with knitted edges or silicone borders

1. Starting from the top, down to half way, turn the sleeve inside out.
2. a Roll the sleeve over your hand and continue up your arm until the rolled-over section reaches your elbow.
b **Compression sleeves with glove:** Slide your hand or finger right into the garment. If the product fits correctly, there should be no creases or gaps between the fingers.
3. Work the rolled section of the sleeve over your elbow and roll it towards your shoulder bit by bit. Spread out the garment's knitted fabric to fit nicely over the entire length of the arm without any creases. If worn correctly, the garment should not restrict your movement. Tip: Before donning, fold over the silicone border (if applicable).

Compression sleeves with shoulder cap and strap or bra fastening

4. Connect the strap ends so that you can afterwards place the loop over your head and arm without the sleeve. Caution: You can change the length of the shoulder strap using the adjuster if it is too loose or too tight.
5. If there is a bra fastening, thread the touch fastener around the bra strap and close it.

Arm wraps

Slide your arm fully into the garment. Proceed as described in Point 3.

Please note

Only wear compression garments if they have been prescribed by a doctor. Compression garments should only be dispensed by properly trained staff at a medical retailer. Your Juzo compression garments are largely resistant to oils, ointments, skin moisture and other environmental influences. The durability of the knitted fabric may be affected by certain combinations of the above. This is why it is advisable to have your compression garments checked regularly at a medical retailer. For compression garments with a silicone border, please note that the arms should be depilated and free from oils, powder, creams and ointments. Please contact your specialist retailer in the event of damage. Do not repair the product yourself – this may affect its quality and medical effectiveness. For hygiene reasons, this product is intended for one person only.

Washing and care instructions

Please note the care instructions on the textile label sewn into your compression garment or the printed information. Do up any fasteners (if there are any) before washing! We recommend turning compression garments inside out before washing. Wash compression garments separately when washing for the first time (they may lose colour). If possible, your compression garments should be washed every day on a gentle or delicate wash cycle (40°C). Tip: a laundry net provides additional protection for the knitted fabric. Please do not use fabric softener! When hand washing, rinse well and do not wring. We recommend the gentle Juzo special detergent. You can reduce drying time by placing the compression garment on a thick towel after washing, tightly rolling the towel up and firmly pressing out excess moisture. Then hang up the compression garment loosely. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Please use the delicates setting when drying in a tumble dryer (suitable for tumble-drying: see textile label). Juzo compression garments are not suitable for dry cleaning.

Material composition

For precise details, please see the textile label sewn into your compression garment or the printed information on the fabric.

Storage information and shelf life

Store in a dry place and protect from sunlight. Standard products have a maximum shelf life of 36 months. The compression garment then has a usage period of 6 months. The usage period depends on correct handling (e.g. care, donning and doffing) and normal use of the products. The usage period of the medical product is printed on the box with an hourglass symbol. Custom-made products are for immediate use. Body measurements may change based on the patient's condition and individual circumstances. Custom-made products are therefore designed for a usage period of 6 months. After the 6-month usage period, body measurements must be re-checked by a medical specialist. In the event of significant changes in measurements of the body part that is being treated (e.g. based on therapeutic success, weight change), re-measurement and supply of a new product may be required earlier, before the 6-month usage period comes to an end.

Indications

Lymphoedema, venous congestion, post-traumatic swelling, lipodema

Following burns and for scar treatment, we recommend using our special scar therapy products.

Absolute contraindications

Advanced peripheral arterial disease (if one of these parameters applies: ABPI < 0.5, ankle artery pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO₂ < 20 mmHg instead). Decompensated heart failure (NYHA III + IV), septic phlebitis, phlegmasia cerulea dolens, acute bacterial, viral or allergic inflammation, swelling of the extremities for unknown reasons.

Relative contraindications

Pronounced weeping dermatoses, intolerance to pressure or of any of the components of the product, severe sensory disorders of the extremities, advanced peripheral neuropathy (e.g. diabetes mellitus), primary chronic polyarthritis, peripheral arterial disease (PAD) stage I/II, malignant lymphoedema.

Important

If there are open wounds in the area to be treated, the affected area must be covered with an appropriate wound dressing or bandage before the compression garment can be applied. Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your symptoms worsen during the time you are wearing the product, please take off the product and consult a medical specialist immediately. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

As a rule, the product is to be destroyed via the conventional disposal route after the end of use. Please observe the applicable local/national regulations.

Additional information for products containing silver

As silver can deduct heat and electricity, please avoid the direct contact with electric sources. Juzo Expert Silver / Juzo Expert Strong Silver / Juzo Dynamic Silver should not come in contact with metals (e.g. metal clothes hangers) when the textile is wet, the worked in silver could be discoloured thereby.

Additional information about the Dip Dye Collection

Juzo Compression Stockings and Compression Sleeves in Dip Dye design are unique products. The colour gradient of the individual items is unique and cannot be reproduced exactly. Owing to the manufacturing process, slight colour variations are possible.

Additional information about the Batik Collection

Juzo compression stockings, compression pantyhose and compression sleeves in tie-dye design are unique products. The colour pattern of the handmade individual items is unique and cannot be reproduced. There may be minor colour changes after the first wash. The characteristic "wrinkled look" originates from the production technique. It is only visible on the unworn product. Due to the special dyeing technique the compression stockings, compression pantyhose and compression sleeves are slightly longer in unworn condition than products without tie-dye design. However, this does not have any influence on the fitting accuracy and the medical efficacy when worn.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany · juzo.com