SMARTCY[®]
SAVER

SMARTCY[®]
SAVERPLUS

SMARTCY[®]
SAVERGEO

EN
PT
HU
Rev. A0/2021

WITHIN
EVERYONES'S
REACH

AED
DAE
DEA/DESA

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**AED
DAE
DEA**

The device has been designed for use by lay personnel as well as by healthcare personnel who have duly completed and passed a BLS course in accordance with international guidelines.

Use it if the patient:

- **Is unconscious and**
- **is not breathing and**
- **has no heartbeat**

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Being a life-saving device, the **AED** must always be available and ready to use; therefore the battery must always be connected to the device.

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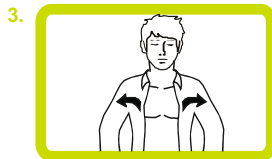
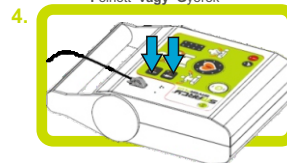
Plug in the PADs connector
and select the patient

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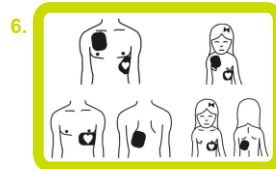
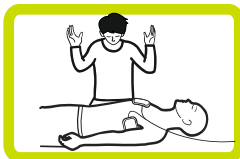
Practice 2 insufflations
Pratique 2 insuflações
2 levegő befújás következik

10.

The Fully Automatic model will deliver the shock automatically, without pressing the shock button, after a warning voice message.

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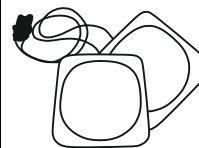
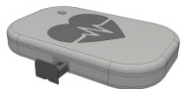
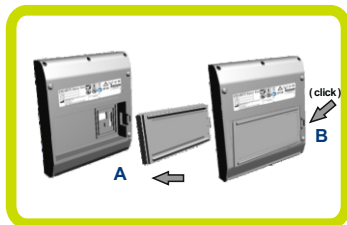
2.

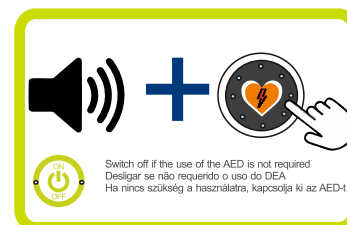
3

Start of the activation test, follow the voice messages provided by the AED

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3.

For the automatic model, pressing the shock button is not required, follow the AED's voice instructions

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Nr.	Function	Nr.	Function
1	Pediatric mode selector To select pediatric patient to treat with universal PADs	9	Red control LED In stand-by mode: error status
2	Adult mode selector To select adult patient to treat with universal PADs	10	PADs connector port
3	Icon "Place the PADS" Place the PADS on the patient's chest	11	Smarty Saver microphone
4	Icon "Don't touch" Do not touch the patient	12	Smarty Saver speaker
5	Shock button To deliver the shock	13	Battery compartment
6	Icon CPR Start the cardiopulmonary resuscitation	14	Contact pins
7	ON/OFF button To switch ON/OFF the device	15	SD Memory Card and USB slot
8	Green control LED In stand-by mode: status ready to use		

In the automatic model, the shock button is replaced by the "auto shock" icon which announces the imminent automatic discharge.

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SMARTCY[®] SAVER PLUS

Smarty Saver Plus Device Structure

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Nr.	Function	Nr.	Function
1	Pediatric mode selector To select pediatric patient to treat with universal PADS	10	Red control LED In stand-by mode: error status
2	Adult mode selector To select adult patient to treat with universal PADS	11	*LED "Connection with CPR Quality sensor" If on, indicates that Bluetooth connection with Q-CPR module is activated
3	Icon "Place the PADS" Place the PADS on the patient's chest	12	PADs connector port
4	*CPR LED bar See imagine "CPR quality feedback"	13	Smarty Saver Plus microphone
5	Icon "Don't touch" Do not touch the patient	14	Smarty Saver Plus speaker
6	Shock button To deliver the shock	15	Battery compartment
7	Icon CPR Start the cardiopulmonary resuscitation	16	Contact pins
8	ON/OFF button To switch ON/OFF the device	17	SD Memory Card and USB slot
9	Green control LED In stand-by mode: status ready to use		

In the automatic model, the shock button is replaced by the "auto shock" icon which announces the imminent automatic discharge.

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CPR FEEDBACK

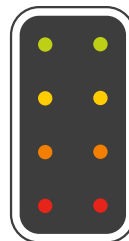
The **Smarty Saver Plus** assists the operator in the correct execution of cardiac massage, during Cardio Pulmonary Resuscitation, thanks to the external CPR Quality sensor (SMT-C14034*).

This external device is in fact able to measure the depth and frequency of the compressions performed and send this feedback to the Smarty Saver Plus device via Bluetooth.

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LED SCALE WITH PROGRESSIVE LIGHTING:

red LED lighting: insufficient or excessive pressure
red and orange LED on: pressure not yet sufficient
red, orange and yellow LED lighting: almost sufficient pressure
red, orange, yellow and green LED lighting: correct pressure

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* see accessories section, CPR Quality Module SMT-C14034; for further information please refer to the specific user manual.

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In the automatic model, the shock button is replaced by the "auto shock" icon which announces the imminent automatic discharge.

* For more information on Geo functionality and CPR Quality feedback, please refer to the specific user manual.

Nr.	Function	Nr.	Function
1	Pediatric mode selector To select pediatric patient to treat with universal PADS	12	* "Key Stream" button To send the ECG streaming to the AMISAVERCLOUD platform
2	Adult mode selector To select adult patient to treat with universal PADS	13	"Remote LED" LED Device in operation: LED on blinking: the ECG trace is being sent LED steady on: Bluetooth connection with the associated Q-CPR module is on Device switching off: LED on blinking: software update in progress*
3	Icon "Place the PADS" Place the PADS on the patient's chest	14	PADS connector port
4	* CPR LED bar See image "CPR quality feedback"	15	Geoloc microphone
5	Icon "Don't touch" Do not touch the patient	16	Smarty Saver Geo microphone
6	Shock button To deliver the shock	17	Geoloc speaker
7	Icon CPR Start the cardiopulmonary resuscitation	18	Smarty Saver Geo speaker
8	ON/OFF button To switch ON/OFF the device	19	SIM card slot
9	Green control LED In stand-by mode: status ready to use	20	Battery compartment
10	Red control LED In stand-by mode: error status	21	Contact pins
11	* VIVO call button To start a live phone call	22	SD Memory Card and USB slot

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CPR Quality Module SMT-C14034 (for Smarty Saver Plus/ Smarty Saver Geo models)

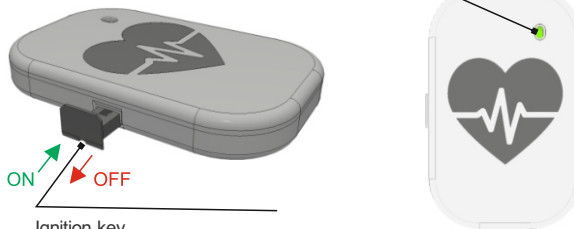
Módulo CPR-Quality SMT-C14034 (somente para modelos Smarty Saver Plus /Smarty Saver Geo)

CPR Quality SMT-C14034 (Smarty Saver Plus /Smarty Saver Geo modellekhez)

Flashing LED: Bluetooth signal search
Fixed LED: Bluetooth connection active

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Ignition key

Chave de acendimento

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External device capable of measuring the depth and frequency of the compressions performed and sending this feedback to the Smarty Saver Plus/Geo device via Bluetooth.

- Turn the module on by pushing the side ignition key
- Place it on the patient's chest at the time of CPR
- Perform compressions by checking that they are correct using the LED bar on the keyboard of the device and with the help of the AED voice instructions

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For all other information regarding the external Q-CPR sensor, please refer to the specific user manual.

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It is advisable to carry out routine maintenance on a daily basis; in the event of any anomalies, contact the service department.

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Thank you for choosing A.M.I. Italia **Smarty Saver Series** defibrillator!

For a correct use of the device, it is necessary, before using it, **to read carefully the relevant user manual (downloadable from www.amiitalia.com) and this user guide.** In order to ensure a correct and rapid traceability of the product and to receive information about all updates implemented, the user is required **to register the device** in the appropriate section of our website www.amiitalia.com.

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